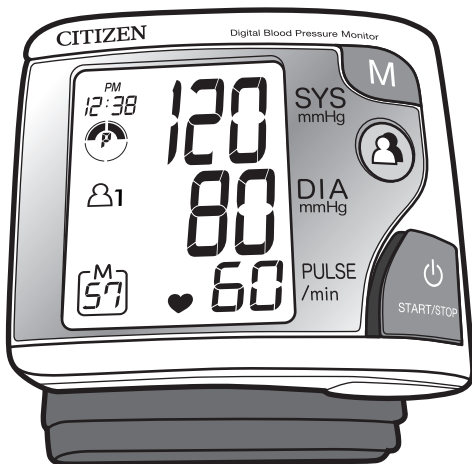
INSTRUCTION MANUAL  
FOR  
DIGITAL BLOOD PRESSURE MONITOR

REF **CH-608**

English  
Español  
Português  
Deutsch  
Italiano  
Русский

## **GB** Contents

- 8- Cautionary Notes
- 23- Nature and frequency of maintenance:  
Protect the nature environment:  
WEEE mark
- 25- Reference European standard:  
European Representative:
- 28- Name/function of each part
- 30- Display explanations
- 32- Hypertension Classification Indicator
- 34- Irregular heartbeat (IHB) indicator
- 36- Applying the cuff
- 38- Correct Measuring Posture
- 40- Loading the batteries
- 43- Removing the batteries
- 45- Time Adjustment
- 47- Measurement Procedures
- 49- Recall Values from Memory
- 51- Clear Values from Memory
- 53- Data Transfer to PC (Optional)
- 55- About Blood Pressure
- 61- Before requesting Repairs or Testing
- 67- Blood Pressure Q&A
- 73- Display Explanations
- 74- SPECIFICATIONS

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## **GB** Cautionary Notes

1. The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
2. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
3. The unit should not be operated by children so to avoid hazardous situations.

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**GB**

4. CH608 is not field serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems, please contact the store where you purchased CH608.
5. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device may have difficulty in determining the proper blood pressure.
6. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash the cuff or use the chemical cleaner on it. Never use thinner, alcohol or petrol(gasoline) as cleaner.

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**GB**

7. To stop inflation at any time, press the  key, and the air in the cuff will be rapidly exhausted.
8. Once the inflation reaches 300mmHg, the unit will start deflating rapidly for safety reason.
9. If the cuff pressure feels abnormal or you experience any other irregularity while using the cuff, reduce the pressure immediately by pressing the  key and then consult the sales outlet where you purchased the monitor.

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**GB**

10. If you think the measurement is abnormal or if measurement makes you feel unwell, discontinue use and consult your physician.
11. Repeated blood pressure measurement may cause problems such as congestion or swelling in some people.
12. Frequently repeated blood pressure measurements will not give accurate results.  
Allow an interval of about 10 minutes between measurements.
13. Do not wrap the cuff around the injured wrist.

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**GB**

14. Be sure to use this unit only for measuring blood pressure. Do not use it for any other purpose.
15. Do not measure your blood pressure after bathing or drinking.
16. Do not try to measure your blood pressure immediately after drinking coffee or tea or after smoking.

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**GB**

17. Your blood pressure varies constantly throughout the day. Ideally, you should measure your blood pressure in the same posture at the same time every day.
18. Do not use the monitor in the vicinity of flammable gases such as those used for anaesthesia. Doing so could ignite the gases and cause an explosion.
19. Do not use the monitor in enriched oxygen environments such as a hospital's hyperbaric chamber or oxygen tent. Doing so could ignite the oxygen and cause a fire.

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**GB**

20. Do not use mobile phones near the monitor as this could result in a malfunction.
21. If you use a cardiac pacemaker, consult your physician before using the monitor.

 0086 The CE marking is meant to provide information to market inspectors in the EU member countries.

 : Type BF applied part

 : Refer to instruction manual before use

 : Appliance compliance WEEE directive

 : Manufacturer

 **Warning**

The device has been tested and homologated in accordance with EN60601-1-2 for EMC. This does not guarantee in any way that the device will not be affected by electromagnetic interference. Avoid using the device in high electromagnetic environment.

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**GB**

### **Nature and frequency of maintenance:**

This product is designed for use over an extended period of time; however, it is generally recommended that it be inspected every two years to ensure proper function and performance.

### **Protect the nature environment:**

Please help to protect natural environment by respecting national and/or local recycling regulations when disposing of the battery and the product at the end of their useful life.

### **WEEE MARK**

If you want to dispose this product, do not mix with general household waste. There is a separate collection systems for used electronics products in accordance with legislation under the WEEE Directive (Directive 2002/96/EC) and is effective only within European Union.

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**GB**

**Reference European standard:**

The blood pressure measuring device corresponds to regulation EN60601-1-2.

**Calibration:**

The blood pressure measuring device is generally recommended to have the monitor inspected every two years to ensure correct functioning and accuracy. Please contact a distributor.

Name : CITIZEN SYSTEMS(H.K.) LIMITED  
Address : FLAT A-B, 4TH FLOOR, HUNG MOU INDUSTRIAL  
BUILDING, 62 HUNG TO ROAD, KWUN TONG,  
KOWLOON, HONG KONG

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Nome : CITIZEN SYSTEMS(H.K.) LIMITED  
Endereço : FLAT A-B, 4TH FLOOR, HUNG MOU INDUSTRIAL  
BUILDING, 62 HUNG TO ROAD, KWUN TONG,  
KOWLOON, HONG KONG

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Name : CITIZEN SYSTEMS(H.K.) LIMITED  
Anschrift : FLAT A-B, 4TH FLOOR, HUNG MOU INDUSTRIAL  
BUILDING, 62 HUNG TO ROAD, KWUN TONG,  
KOWLOON, HONG KONG

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Nome : CITIZEN SYSTEMS(H.K.) LIMITED  
Indirizzo : FLAT A-B, 4TH FLOOR, HUNG MOU INDUSTRIAL  
BUILDING, 62 HUNG TO ROAD, KWUN TONG,  
KOWLOON, HONG KONG

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
Наименование : CITIZEN SYSTEMS(H.K.) LIMITED  
Адрес : FLAT A-B, 4TH FLOOR, HUNG MOU INDUSTRIAL  
BUILDING, 62 HUNG TO ROAD, KWUN TONG,  
KOWLOON, HONG KONG

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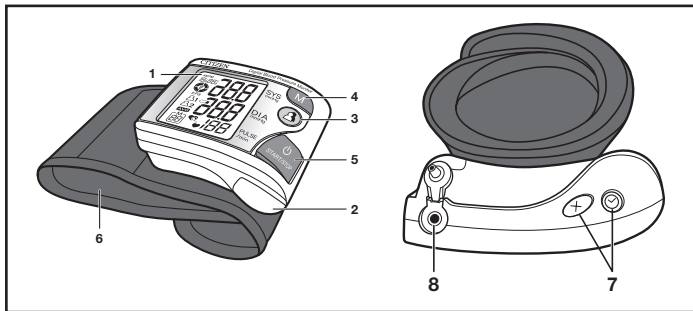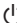
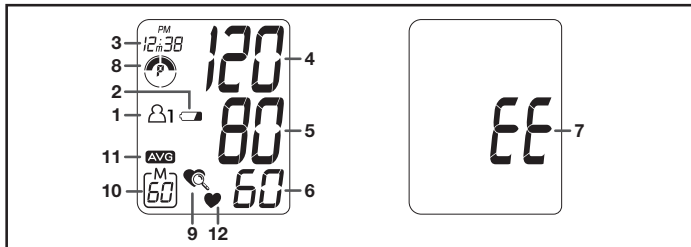
## **GB** Display Explanations

1. Memory Zones
2. Weak Battery Mark
3. Date/Time Indicator
4. Systolic Pressure
5. Diastolic Pressure
6. Pulse Rate
7. Error Mark
8. Hypertension Classification Indicator
9. Irregular Heartbeat (IHB) Indicator
10. Memory Sequence Number
11. Memory Average
12. Pulse Symbol

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**Stage 1 Hypertension**

**Normal**

## **GB** Hypertension Classification Indicator

The National High Blood Pressure Education Program Coordinating Committee has developed a blood pressure standard according to which areas of low- and high-risk blood pressure are identified, classifying blood pressure ranges into 4 stages. This unit is equipped with innovative blood pressure classification indicator, which visually indicates the assumed risk level (prehypertension / stage 1 hypertension / stage 2 hypertension) of the result after each measurement. (Please note above.)

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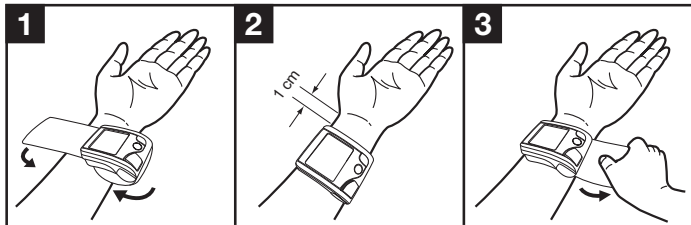
### **GB** Applying the cuff

Apply cuff to left wrist with palm facing up (1). Make sure the edge of the cuff is about 1 cm from the palm (2). Fasten the velcro strap securely around your wrist (3).

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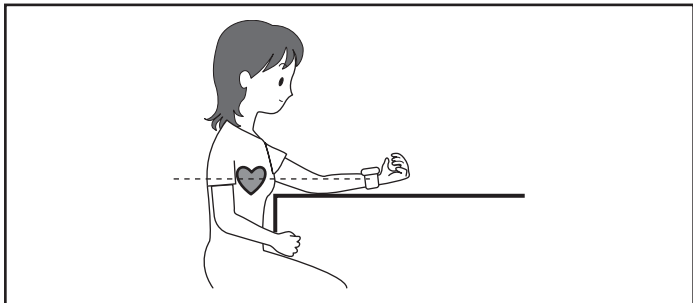
**GB Correct Measuring Posture**

Place your elbow on the table so that the cuff is at the same level as your heart.

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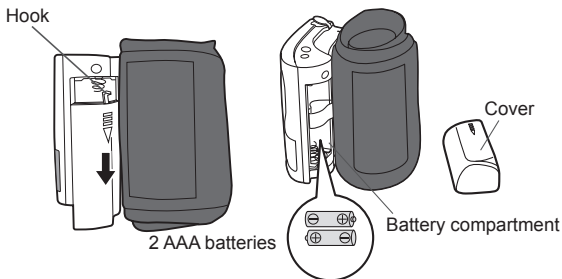
## **GB** LOADING THE BATTERIES

(Batteries supplied with the unit should be loaded in compartment before putting the Blood pressure monitor in use.)

### **1** Open the cover of battery compartment.

Pressing the hook down, pull the cover toward you to open.


\* If the Blood pressure monitor is left unused for an extended period of time, the batteries should be removed from the compartment.

### **2** Place batteries in the compartment.

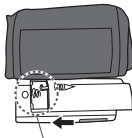Put the batteries over the battery-pull strap, paying attention to the positive and negative terminal symbols  $\oplus$  and  $\ominus$ .

### **3** Close the battery compartment cover.

Fitting the lugs of the cover at the mating recesses of the battery compartment body, push the hook to close the cover.

Fit the lugs (projections) at the recesses.

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## **GB REMOVING THE BATTERIES**

- When the icon  is displayed, or where the clock is no longer shown or nothing appears in the display, replace the old batteries with two new ones. All two batteries should be changed at once.
- Stored data will not be erased when batteries are changed, while time information will be erased. Make sure to adjust the time setting after changing batteries.
- Do not use rechargeable batteries.
- Do not use alkaline and manganese batteries together.
- The monitor is powered by the batteries provided. Note that they may not last as long as indicated.
- When disposing of used batteries, comply with governmental regulations or environmental public institution's rules that apply in your country /area.

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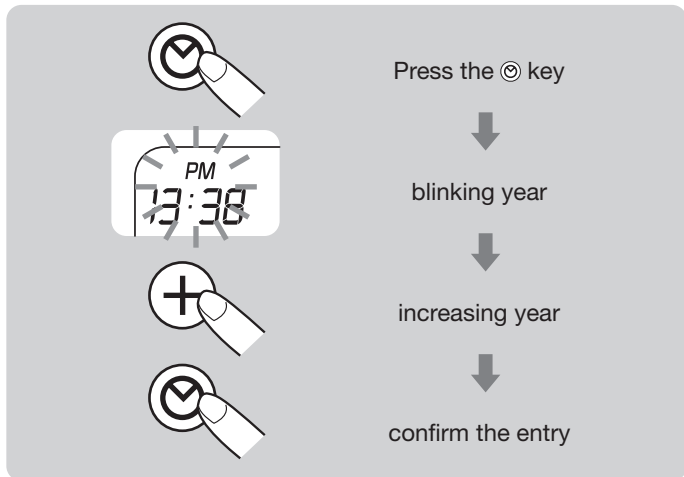
### **GB** Time Adjustment

Press the “” key and “” to adjust the year. Then press the “” and “” in turn to adjust the month, date, hour, minute and confirm the entry.

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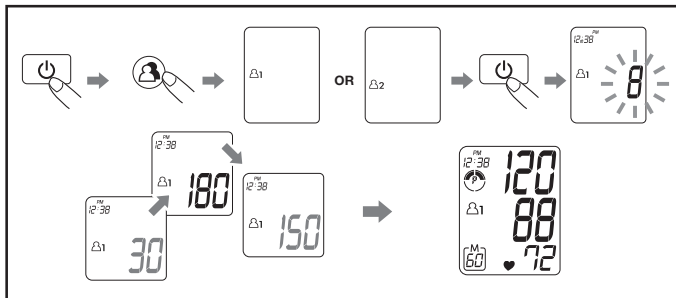
## **GB** Measurement Procedures

- 1. Press the “ $\text{⏻}$ ” key.**
- Press the “User-Switching” key to select a memory zone after seeing the display with all digits lighting up.
- 3. Press the “ $\text{⏻}$ ” key again, and the monitor will inflate.**
- When completed, the systolic pressure, diastolic pressure and pulse rate will be shown simultaneously on the same LCD screen. The values are automatically stored in the pre-designated memory zone.

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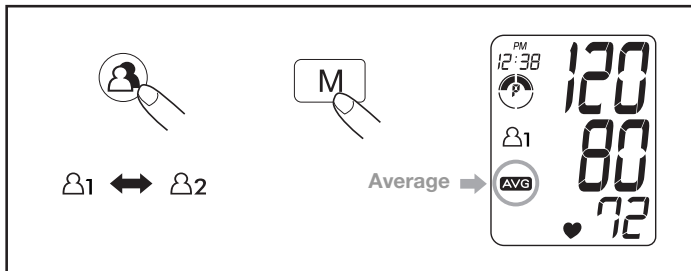
### **GB** Recall Values from Memory

Press the “User-Switching key” to select a memory zone, and press the “M” key to switch from memory to memory with sequence numbers. (The average of the previous measurements will be displayed first. The calculated average will be applied to a maximum of the last 3 memories.)

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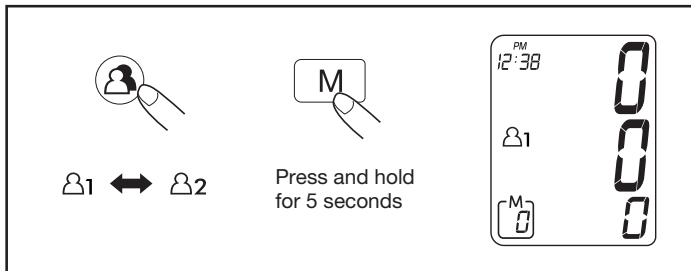
### **GB** Clear Values from Memory

Press the “User-Switching key” to select a memory zone, then press and hold the “M” key for approximately 5 seconds to clear the data in the pre-designated memory zone.

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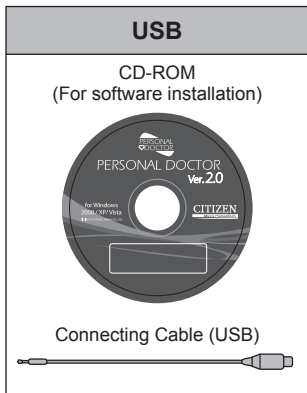
### **GB Data Transfer to PC (Optional)**

**Accessories for data transfer to PC including :**

1. Personal Doctor (PD) software CD-ROM
  2. connecting cable (USB)
  3. instruction manual
- are available in another separate pack for option.

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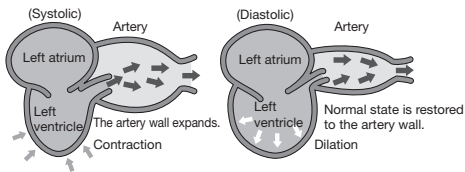
## **GB** ABOUT “BLOOD PRESSURE”


### **What is blood pressure?**

The heart is a pump that circulates blood throughout your body. Blood is pumped from the heart at a constant pressure into arteries. This pressure is called the arterial blood pressure and represents, in general terms, your blood pressure. Blood pressure is indicated by several kinds of pressures, including the systolic pressure that occurs when the heart pumps blood and the diastolic pressure that occurs when blood returns to the heart.

### **Your blood pressure changes all the time.**

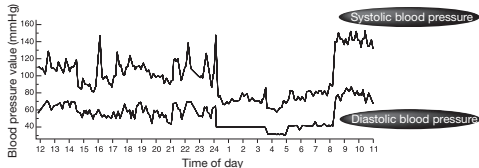Your blood pressure differs according to your age, gender and a range of other factors. It is likely to be affected by your biorhythm during the day and by your posture, physical activities, mental activities, level of stress and even by the air temperature. The blood pressure of a healthy person generally varies within a day.

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## **GB** Before requesting Repairs or Testing

Make sure to check the following before sending your blood pressure monitor away to be tested or repaired.

Problem	Checks	Response
Display stays blank when you press the  key.	Check whether the batteries are dead.	Replace all the batteries with new ones.
	Check whether the batteries are installed the right way round ( $\oplus$ and $\ominus$ oriented correctly).	Load the batteries in the correct direction.
No measurements can be taken.	Check whether the measurement icon  is displayed.	Attach the cuff properly.
	Check whether the cuff is applied correctly.	
	Did you stay calm during measurement?	Measure your blood pressure again, making sure to remain still.
	Note that blood pressure measurement may not be possible for someone with an extremely weak pulse or cardiac arrhythmia (irregular pulses).	
Your blood pressure readings are abnormally high or low compared with those taken in hospital.		 See page 67.
Other phenomena		Remove the batteries from the unit and change them with new ones.

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## **GB** BLOOD PRESSURE Q&A

**Q** Why is the blood pressure measured at home different from that measured by the physician or at a hospital?

**A** Your blood pressure may vary due to exercise, the ambient temperature or your mental state. When you have a physician or nurse measure your blood pressure, the pressure value is likely to be 10 to 20 mmHg higher than usual due to anxiety and/or stress. Knowing what affects your blood pressure helps you to better monitor your health.

**Q** Why does the blood pressure obtained vary with measurement?

**A** Our blood pressure is adjusted by the function of automatic nerve. Blood pressure differs with every heart beat. We are likely to think that our blood pressure is constant, but it varies if you take measurements in repetition. Blood pressure is susceptible to the time of day, such as forenoon and afternoon, season and atmospheric temperature. In addition, blood pressure is easily affected by mental stress or emotional ups and downs. It tends to increase when you feel tension or decrease when you are relaxed.

**Q** What is the benefit of measuring my blood pressure at home?

**A** Blood pressure measurements taken at home give reliable data as they can be taken when you are in a stable and relaxed condition. Physicians place considerable importance on blood pressure measurements taken at home. You should measure your blood pressure at the same time each day and record the variations in your blood pressure.

Always consult your physician for an interpretation of your blood pressure readings and to determine the proper treatment.

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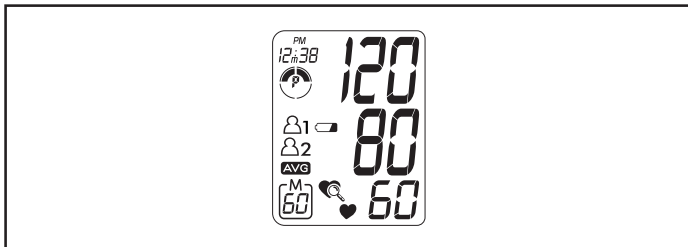
### Display:

### Icons:

	User 1: Appears when the monitor is operated by User 1.
	User 2: Appears when the monitor is operated by User 2.
	Weak Battery Symbol: Appears when batteries should be replaced.
	Pulse Symbol: Shows the pulse rate per minute.
	Irregular Heartbeat (IHB) indicator
<b>M</b>	Memory Symbol: Shows the memory reading number.
<b>AVG</b>	Memory Average: Displays average of last 3 readings.
	Hypertension Classification Indicator: Compares blood pressure against guidelines. See page 32 for more information.

<b>EE</b>	Measurement Error: Measure again. Wrap the cuff correctly and keep wrist steady during measurement.
<b>E 1</b>	Air Circuit Abnormality: Measure again.
<b>E2</b>	Pressure Exceeding 300 mmHg: Switch the unit off clear, then measure again.
<b>E3</b>	Data Error: Remove and reload the batteries.
<b>E<sub>r</sub></b>	Exceeding Measurement Range: Measure again quietly.

\* If error keeps occurring, return the device to the local distributor or service center.

## GB SPECIFICATIONS

Model Number:		CH-608	
Measurement system:		Oscillometric method	
Measurement Localization		Wrist	
Cuff:		Pre-formed cuff	
Wrist circumference range:		13.5 ~ 22.0 cm	
Measurement range:	Pressure	0 to 299 mmHg	
	Pulse	40 to 199 pulse/min	
Accuracy:	Pressure	±3 mmHg	
	Pulse	±5% of reading	
	Clock	±30 second/Month	
LCD displays:	Pressure	3 digits	
	Pulse	3 digits	
	Date	4 digits	
	Time	4 digits with AM/PM mark	
	Icons	<b>M</b> : Memory icon, <b>♥</b> : Pulse icon, <b>U1, U2</b> : USER, <b>AVG</b> : Memory average icon, <b>H</b> : Hypertension icon, <b>B</b> : Battery icon, <b>IHB</b> : IHB icon	
Switch:	5 (U, U, M, H, I)		
Inflation:	Automatic Inflation by internal pump		
Deflation & Exhaust:	Automatic constant deflation & Quick		
Rated voltage:	3V  3W ( : direct current)		
Power supply:	2AAA batteries (LR03)		
Battery duration:	Alkaline	Approx. 300 times	(170 mmHg, once/day, 22°C)
Automatic changeover function:	Approx. 1 min. (after activated)		
Memory capacity:	60 sets of data including date and time (for each zone x 2)		
Data output:	PC-link by USB		
Dimensions:	78(W) x 66(H) x 32(D) mm (not including cuff)		
Weight:	Approx. 115g w/o batteries		
Operating conditions:	Temperature	10°C to 40°C	
	Humidity	30% to 85% RH	
Storage conditions:	Temperature	-10°C to 60°C	
	Humidity	30% to 90%RH	
Electric shock protection:	Internal power unit		
Safety Classification:	Type BF Equipment		
Accessories	2AAA batteries for the monitor (LR03), Instruction Manual, Storage case		

- CITIZEN is a registered trademark of Citizen Holdings Co., Japan.  
CITIZEN es una marca registrada de Citizen Holdings Co., Japón.
- Design and Specification are subject to change without notice.

---

## **CITIZEN SYSTEMS JAPAN CO., LTD.**

6-1-12, Tanashi-cho, Nishi-Tokyo-Shi,  
Tokyo 188-8511, Japan

E-mail: [sales-oe@systems.citizen.co.jp](mailto:sales-oe@systems.citizen.co.jp)

<http://www.citizen-systems.co.jp/>