
CITIZEN
Micro HumanTech

English

INSTRUCTION MANUAL
FOR
DIGITAL BLOOD PRESSURE MONITOR

CH-452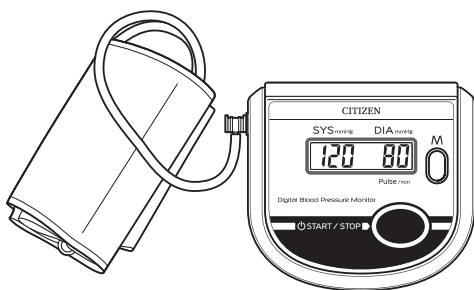


English
Español

1401

Contents

- 3 - GENERAL REMARKS
- 5 - PRECAUTIONS FOR USE AND MAINTENANCE
- 7 - FCC STATEMENT
- 8 - IDENTIFICATION OF PARTS
- 9 - LOADING THE BATTERIES
- 11 - REMOVING THE BATTERIES
- 12 - MEASURING YOUR BLOOD PRESSURE
- 17 - TIPS FOR OBTAINING ACCURATE MEASUREMENTS
- 18 - CALLING UP THE DATA STORED IN MEMORY
- 20 - KEY TO DISPLAY ICONS
- 21 - ABOUT "BLOOD PRESSURE"
- 22 - BLOOD PRESSURE Q&A
- 23 - BEFORE REQUESTING REPAIRS OR TESTING
- 24 - SPECIFICATIONS
- 25 - 1-YEAR LIMITED WARRANTY
- 29 - SPANISH

Ensure that you have all the following components

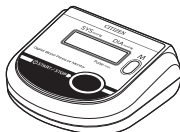Blood pressure
monitor unit

Cuff
(model: SCN-003)

(for monitor)
4 AA-size
batteries


Instruction
Manual

[OPTION]

- Large Adult
Cuff
(model:
SCL-005)

Quick Start Guide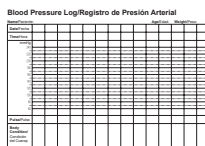

Blood Pressure Log

FAQ's

- Eng 2 -

GENERAL REMARKS

English

General remarks on blood pressure and blood pressure measurement

1. To deflate the cuff quickly, press the "START/STOP" switch.
2. Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:
Cuff fits arm circumference: 8-3/4" - 12-1/2" (22-32 cm)
[OPTION]
Large Adult Cuff fits arm circumference: 12-1/2" - 16-1/2" (32-42 cm)
3. When attaching the cuff, adjust the position of the cuff so that the blue marker is located in the center of your arm and the hem of the cuff is 0.4"-0.8" (1-2 cm) for Cuff / 0.8"-1.2" (2-3 cm) for Large Adult Cuff above your elbow.
4. The monitor is not intended to be a diagnostic device, because it is intended for home use.
5. Self measurement is not therapy! In no event should you change the dosage of your medication prescribed by your physician.
6. As preparation for measuring your blood pressure, you should urinate and then remain relatively still for 10 to 15 minutes prior to measurement.
7. Exercising, eating, drinking, smoking, etc., prior to measurement may affect the results.
8. Your blood pressure varies constantly throughout the day. Measurements should be taken regularly at the same time each day.
9. DO NOT place too much emphasis on the results of one measurement. Keep a continuing record of your blood pressure variations. A complete picture can only be obtained from a large number of readings.
10. Emotional stress tends to cause your blood pressure to rise.


To obtain correct blood pressure measurements

1. Take five or six deep breaths and then relax before measuring your blood pressure. If you are tense when taking the measurement, you will not get a valid reading.
2. Your blood pressure will be elevated if you are anxious or irritated, suffering from lack of sleep or constipation, or have just taken some exercise or eaten a meal.
3. Wrap the cuff around your arm and tighten it so that one finger can be inserted between the cuff and arm. ( See page 12, 13)
4. DO NOT measure your blood pressure after bathing or drinking.
5. If you feel the need to urinate, do so before measuring your blood pressure.
6. Measure your blood pressure where the room temperature is around 68°F (20°C). DO NOT measure your blood pressure when it is too cold (below 50°F (10°C)) or too hot (above 104°F (40°C)) in the room.
7. DO NOT try to measure your blood pressure immediately after drinking coffee or tea or after smoking.

- Eng 3 -

GENERAL REMARKS

8. Measure your blood pressure when you are relaxed and still. Keep the center of the cuff at the level of your heart and DO NOT move your arm or talk.
9. DO NOT measure blood pressure when the cuff has been on your arm for a few minutes or more. During this time, your arm will have built up a higher blood pressure and you will not get valid reading.
10. The figure below shows the blood pressure classifications under the WHO standards.

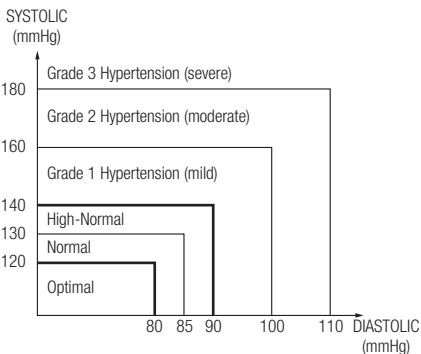**Remarks:**

The graph is not exact, but may be used as a guide in understanding non-invasive blood pressure measurements. The device is only intended for use by adults.

Definitions and Classifications of Blood Pressure Levels

11. Intended Use

- This device is noninvasive blood pressure monitor by oscillometric method. This device can measure the systolic blood pressure (SYS), the diastolic blood pressure (DIA) and the pulse rate automatically.
- This device is intended to be used for checking personal health condition at home under the direction of a physician and is not intended to be a diagnostic device.
- This device is not intended for use with neonates or infant, and not intended for automatic cycling measurement.

Measure your blood pressure at the same time each day.

* Your blood pressure changes all the time. This means that data gathered over a long period has far more significance than data from just one measurement. For this reason, you must measure your blood pressure on a daily basis. Ideally, you should measure your blood pressure at the same time each day, wherever possible.

PRECAUTIONS FOR USE AND MAINTENANCE

English

Precautions for use

1. If you suffer from heart disease, high blood pressure or other circulatory disease, consult your physician before using the monitor.
2. If the cuff pressure feels abnormal or you experience any other irregularity while using the cuff, reduce the pressure immediately by pressing the "START/STOP" switch, detach the cuff or unplug the air hose connector from the monitor and then consult the sales outlet where you purchased the monitor.
3. If you think the measurement is abnormal or if measurement makes you feel unwell, discontinue use and consult your physician.
4. Blood pressure measurement may not be possible for anyone with a weak pulse or arrhythmia.
5. Repeated blood pressure measurement may cause problems such as congestion or swelling in some people.
6. Frequently repeated blood pressure measurements will not give accurate results. Allow an interval of about 1 minute between measurements.
7. If you suffer from a severe problem with blood circulation in your arms, consult your physician before using the monitor. Failure to do so could be hazardous to your health.
8. Measurement may not be possible for anyone with insufficient blood flow to the area where measurements will be taken or who suffers from a frequent irregular heartbeat. Consult your physician for advice on whether to use the monitor.
9. DO NOT wrap the cuff around an injured arm.
10. DO NOT wrap the cuff around an arm in which a drip (intravenous infusion) is inserted, or which is being used for blood transfusion as part of medical treatment. Doing so could result in an injury or a serious accident.
11. DO NOT use the monitor in the vicinity of flammable gases such as those used for anaesthesia. Doing so could ignite the gases and cause an explosion.
12. DO NOT use the monitor in enriched oxygen environments such as a hospital's hyperbaric chamber or oxygen tent. Doing so could ignite the oxygen and cause a fire.
13. DO NOT use mobile phones near the monitor as this could result in a malfunction.
14. If you use a cardiac pacemaker, consult your physician before using the monitor.
15. Be sure to use this unit only for measuring blood pressure. DO NOT use it for any other purpose.
16. Be sure to use only our authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.
17. DO NOT use this unit on infants.
18. Blood pressure measurement may not be possible for anyone with common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation.

- Eng 5 -

PRECAUTIONS FOR USE AND MAINTENANCE**Maintenance Precautions**

1. DO NOT store the blood pressure monitor in locations exposed to direct sunlight, high temperatures (over 140°F (60°C)), low temperatures (below -4°F (-20°C)), high relative humidity (over 95%) or excessive amounts of dust. Make sure to store the blood pressure monitor, where children, pets and or pests are not there.
2. DO NOT drop the blood pressure monitor or subject it to other shocks or vibration.
3. Remove the batteries if the monitor will be left unused for a long period.
4. DO NOT attempt to disassemble the monitor.
5. DO NOT bend the cuff or the air hose excessively.
6. If the monitor is very dirty, wipe it clean with a cloth moistened with sterilizing alcohol or a neutral detergent. Then wipe it with a dry cloth.
7. NEVER clean the blood pressure monitor with thinners or benzene, as this could damage the monitor.
8. To clean the cuff, wipe it with a moist cloth. Avoid hard rubbing, as this will cause air leakages. Take care also not to get water into the air hose.

**Warning**

No modification of this device is allowed.

**Warning**

The device has been tested and homologated in accordance with EN60601-1-2 for EMC. This does not guarantee in any way that the device will not be affected by electromagnetic interference. Avoid using the device in high electromagnetic environment.

Distributed by:

CITIZEN SYSTEMS AMERICA CORP.

363 Van Ness Way, Suite 404 Torrance, CA 90501

<http://www.citizen-systems.com>

TOLL FREE / 800-421-6516

FCC STATEMENT

NOTE:

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled "Digital Apparatus", ICES-003 of the Canadian Department of Communications.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.


English

- Eng 7 -


IDENTIFICATION OF PARTS

Component names

<Main Unit>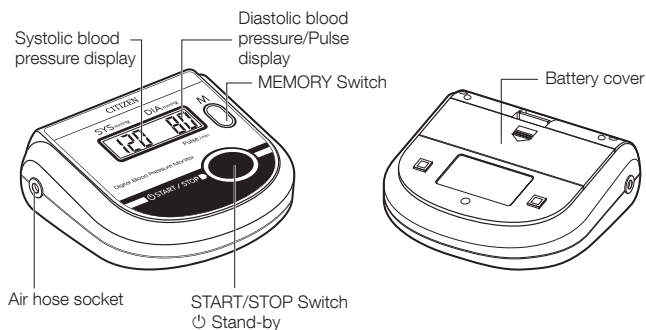

<Cuff>

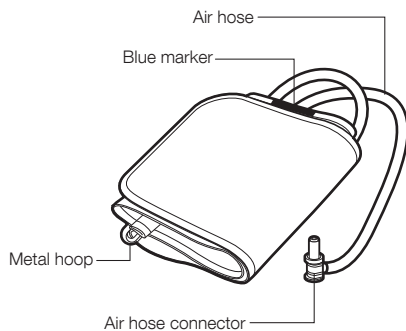
LOADING THE BATTERIES

(Batteries supplied with the unit should be loaded in compartment before you use the blood pressure monitor.)

1 Open the cover of battery compartment.

Pressing the hook down, pull the cover toward you to open.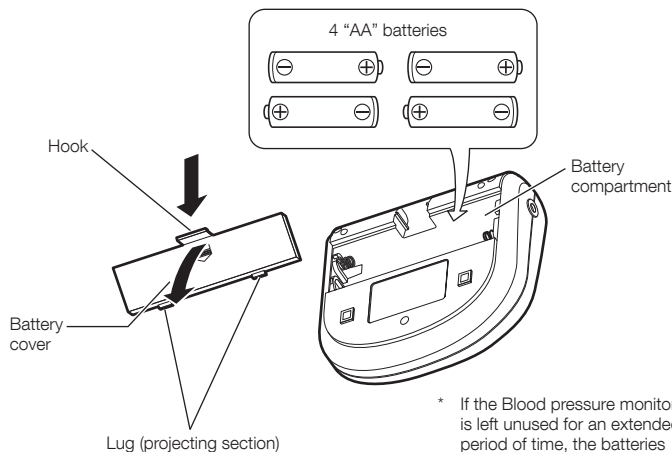


* If the Blood pressure monitor is left unused for an extended period of time, the batteries should be removed from the compartment.

2 Place batteries in the compartment.

Put the batteries paying attention to the positive and negative terminal symbols $\oplus$ and $\ominus$.

LOADING THE BATTERIES**3 Close the battery cover.**

Fitting the lugs of the cover at the mating recesses of the battery compartment body, push the hook to close the cover.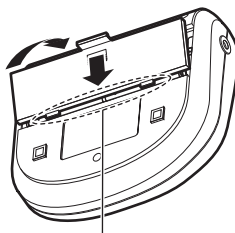

Fit the lugs (projections) at the recesses.

REMOVING THE BATTERIES

- When the icon  is displayed or nothing appears in the display, replace the old batteries with four new ones. All four batteries should be changed at once.
- The memory is erased if the batteries are removed for replacement. It is recommended to make note of the data stored in memory prior to changing batteries.
- Do not use rechargeable batteries.
- Do not use alkaline (LR6) and manganese (R6P) batteries together.
- Batteries included are for demonstration purposes only. Battery life may be shorter than specified.
- When disposing of used batteries, comply with governmental regulations or environmental public institution's rules that apply in your country/area.

English

MEASURING YOUR BLOOD PRESSURE

1 ATTACHING THE CUFF

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:


Cuff fits arm circumference: 8-3/4" - 12-1/1" (22-32 cm)

[OPTION]


Large Adult Cuff fits arm circumference: 12-1/2" - 16-1/2" (32-42 cm)

- 1 Unroll the cuff and thread the end through the metal hoop so that the side with the hook-and-loop fastener is on the outside.

- 2 Put the cuff around your arm so that the blue marker comes to the palm side.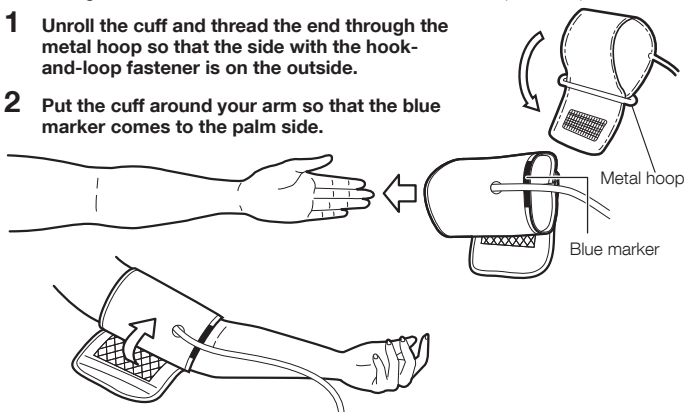

- 3 Adjust the position of the cuff so that the blue marker is located in the center of your arm and the hem of the cuff is 0.4"-0.8" (1-2 cm) for Cuff/0.8"-1.2" (2-3 cm) for Large Adult Cuff above your elbow.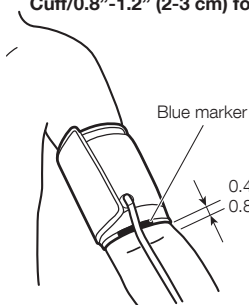


- * The cuff should be put on the bare arm or over a light-weight underwear.
- * If you wear a heavy-weight top, please remove it.

0.4"-0.8" (1-2 cm) for Cuff
0.8"-1.2" (2-3 cm) for Large Adult Cuff


MEASURING YOUR BLOOD PRESSURE

English


- 4** Pull the end of the cuff outwards so that the cuff is snug around your arm and then secure the end to the hook-and-loop fastener.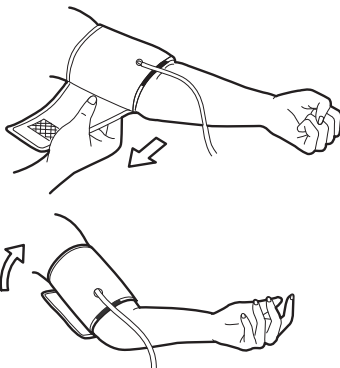

- 5** Place your arm on a table or the like, so that the center of the cuff comes to the height of your heart.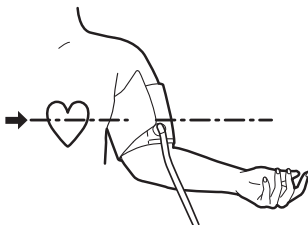

* Carefully place the cuff on your arm, taking care to use the correct tightness. The tightness is correct if you can readily slide a finger between the cuff and your arm.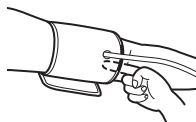


- 6** Relax your arm and lightly open your hand, with the palm upward.

MEASURING YOUR BLOOD PRESSURE

2 Plug the air hose connector into the main unit.

- Plug the air hose connector firmly into the air hose socket on the main unit, as illustrated.

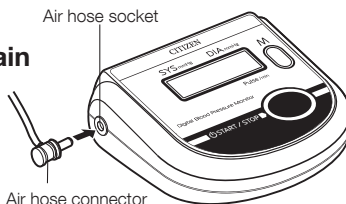3 Place the cuff at the height of the heart.

4 Press the START/STOP switch (the blue switch).

* If the pressurization value exceeds 280 mmHg, if pressurization feels abnormal, or if you want to stop the measurement process, press the START/STOP switch again. The cuff deflates, and the power turns off.

- ① All digits on the display blink for approximately one second.
- All digits displayed is an initial display, indicating that the monitor is functioning normal.
The **BT** mark will also be displayed but this does not indicate that the battery needs to be replaced.
- ② ♥ is displayed for approximately one second.
- ③ The cuff is pressurized automatically.

MEASURING YOUR BLOOD PRESSURE

English

- ④ ♥ is displayed soon after pressurization is complete, and measurement begins automatically.
* If the unit judges that pressurization is insufficient, it automatically repressurizes.


- ⑤ ♥ blinks each time a pulse is detected, and the pressure value is displayed on the diastolic blood pressure display.

- ⑥ Once measurement is complete, the cuff deflates and the measurement results (systolic/diastolic blood pressure values, pulse) are displayed.
* Systolic/diastolic blood pressure values (two seconds) and pulse (one second) are displayed alternately.
* Provided there is no error in the measurement results, they are stored automatically.

5 Finishing measurement.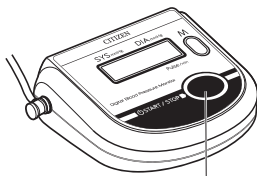


START/STOP Switch

- Press the "START/STOP" switch to turn the monitor off.
- * When you want to stop measurement, press the "START/STOP" switch. The cuff is deflated and measurement stops.

MEASURING YOUR BLOOD PRESSURE

[Manual pressurization]

- If your systolic blood pressure is expected to exceed 200 mmHg, use Manual Pressurization when taking a measurement.
- Keep "START/STOP" switch held pressed until the pressure value you want to stop pressurization is reached (approximately 40 mmHg higher than the Systolic blood pressure). You can stop pressurization by the unit by releasing the switch at the aforementioned value is reached.
The upper pressure limit is 280 mmHg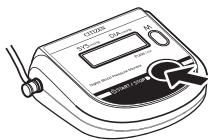


AUTOMATIC POWER OFF FEATURE

- After taking the blood pressure measurement, pressing the "START/STOP" switch will turn the monitor off. However, if you do not press the switch, the monitor's Auto OFF function will automatically turn the monitor off after 3 minutes.

TIPS FOR OBTAINING ACCURATE MEASUREMENTS


Your blood pressure varies according to your posture, the time of day and a range of other factors. Ideally, you should measure your blood pressure in the same posture at the same time every day.

■ Taking measurements while sitting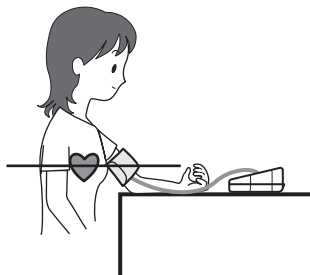

- ① Place your arm on a table or similar surface with your forearm extended.
- ② It is important to have the cuff level with your heart.
- ③ Open your hand slightly on the table so that your palm is facing up and your fingers are relaxed.
- ④ Do not move your body or talk while taking the measurement.

■ Measurement in a reclining posture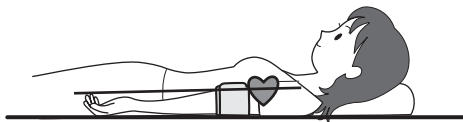


* Relax yourself to avoid placing pressure on the cuff.

- ① Lie down, facing the ceiling.
- ② Lightly open your hand with palm up and straighten your arm.
- ③ Relax your body, arm and fingers.
- ④ Do not move your body or talk during measurement.


■ You may use your right arm to measure your blood pressure.

Blood pressure value is likely to differ by as much as 10 mmHg when measured on the right arm instead of the left (or vice-versa). Measure your blood pressure on the same arm each day.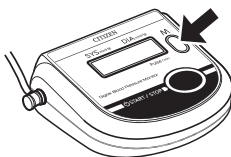


- 90 measurement results can be stored in the memory.
- Except for the case where the unit is engaged in measurement, press "MEMORY" switch to call up the values (measurement results) stored in memory to the display.
- * To erase all the data, remove the batteries. All the stored data is erased.

HOW TO CALL UP THE DATA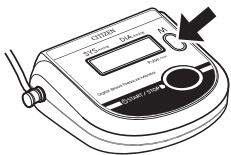

- Press the "MEMORY" switch.

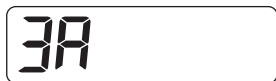- "3A" indicating the average value appears on the left section of the display. Then, the average value, based on the last three measurements, is displayed. ("3A" and the average value do not appear if there are two or fewer measurements stored in memory.)
- Press the "MEMORY" switch again. This recalls and displays first the memory number and then the last set of data stored.

CALLING UP THE DATA STORED IN MEMORY

English

1

120 77

2

130 80

- Press the “MEMORY” switch again. This recalls and displays the second last set of data stored. Each time you press this switch, the unit displays the data prior to the last set of data displayed.

- The value stored in memory is numbered in the order of measurements. For example, when 90 sets of data are stored in memory, the data number 1 represents the latest data.

Data number: 1, 2, 3, 89, 90

↑
The latest
data

↑
The earliest
data

- * You can fast-forward the data numbers in display by pressing and holding the “MEMORY” switch.

KEY TO DISPLAY ICONS

Icon displayed	Condition/Cause	
	Lit	Measurement is under way.
	Blinking	Pulse detected.
	"PUL Err" is displayed when the pulse value is below 40 or above 181.	

If any of the following marks is displayed, take the specified action, then repeat the operation.


Icon displayed	Condition/Cause	Corrective action
 	Appears when the blood-pressure could not be measured accurately.	Apply cuff properly and measure again.
	The pressure is 281 mmHg or higher.	If the pressure does not automatically decline during measurement, immediately press the "START/STOP" switch and turn the unit off. Take the measurement again.
	Appears when the batteries are running low.	Replace the batteries.
	Detected problem in the device.	Contact the vendor.

ABOUT “BLOOD PRESSURE”

English

What is blood pressure?


The heart is a pump that circulates blood throughout your body. Blood is pumped from the heart at a constant pressure into arteries. This pressure is called the arterial blood pressure and represents, in general terms, your blood pressure. Blood pressure is indicated by several kinds of pressures, including the systolic pressure that occurs when the heart pumps blood, and the diastolic pressure that occurs when blood returns to the heart.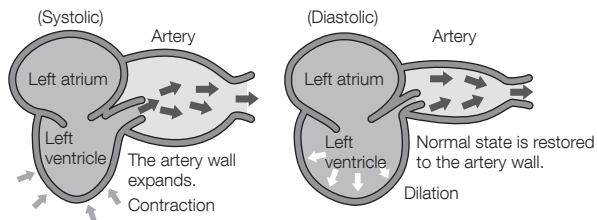

Your blood pressure changes all the time.

Your blood pressure differs according to your age, gender and a range of other factors. It is likely to be affected by your biorhythm during the day and by your posture, physical activities, mental activities, level of stress and even by the air temperature. The blood pressure of a healthy person generally varies within a day.

[Example of blood pressure variations within a day]

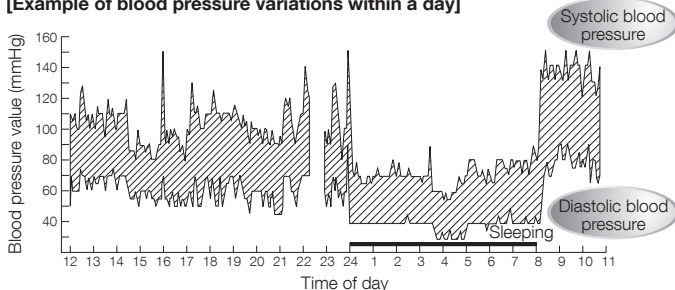(Blood pressure values measured at five-minute intervals through the day)

● Bevan AT, Honour AJ, Stott FH. Clin Sci 1969;36:329-44.

- Eng 21 -

BLOOD PRESSURE Q&A

- Q** Why is the blood pressure measured at home different from that measured by the physician or at a hospital?
- A** Your blood pressure may vary due to exercise, the ambient temperature or your mental state. When you have a physician or nurse measure your blood pressure, the pressure value is likely to be 10 to 20 mm-Hg higher than usual due to anxiety and/or stress. Knowing what affects your blood pressure helps you to better monitor your health.
- Q** Why does the blood pressure obtained vary with measurement?
- A** Our blood pressure is adjusted by automatic nerve function. Blood pressure differs with every heart beat. We are likely to think that our blood pressure is constant, but it varies if you take measurements in repetition. Blood pressure is susceptible to the time of day, such as morning and afternoon, season and atmospheric temperature. In addition, blood pressure is easily affected by mental stress or emotional ups and downs. It tends to increase when you feel tension or decrease when you are relaxed.
- Q** What is the benefit of measuring my blood pressure at home?
- A** Blood pressure measurements taken at home give reliable data as they can be taken when you are in a stable and relaxed condition. Physicians place considerable importance on blood pressure measurements taken at home. You should measure your blood pressure at the same time each day and record the variations in your blood pressure.

Always consult your physician for an interpretation of your blood pressure readings and to determine the proper treatment.

- Eng 22 -

BEFORE REQUESTING REPAIRS OR TESTING

Make sure to check the following before sending your blood pressure monitor away to be tested or repaired.

English

Problem	Checks	Response
Display stays blank when you press the "START/STOP" switch.	Check whether the batteries are dead.	Replace all the batteries with new ones.
	Check whether the batteries are installed the right way round (⊕ and ⊖ oriented correctly).	Load the batteries in the correct direction.
No measurements can be taken.	Check whether the measurement icon ♥ is displayed.	Attach the cuff properly.
	Check whether the cuff is attached correctly.	
	Did you stay calm during measurement?	Measure your blood pressure again, making sure to remain still.
	Note that blood pressure measurement may not be possible for someone with an extremely weak pulse or cardiac arrhythmia (irregular pulse).	
Your blood pressure readings are abnormally high or low compared with those taken in hospital.		 See page 22.
Other phenomena		Remove the batteries from the unit and change them with new ones.

SPECIFICATIONS

Model Number:		CH-452
Measurement system:		Oscillometric method
Measurement Localization		Upper Arm
Cuff:		Soft cuff
Adult cuff circumference range:		8-3/4" - 12-1/2" (22.0-32.0 cm)
Measurement range:	Pressure	0 to 280 mmHg
	Pulse	40 to 180 pulse/min
Accuracy:	Pressure	±3 mmHg
	Pulse	±5% of reading
LCD displays:	Pressure	3 digits
	Pulse	3 digits
	Icons	♥: Measurement icon,  : Battery icon
Switch:		2 (START/STOP, MEMORY)
Inflation:		Automatic inflation by internal pump
Deflation:		Automatic speed deflation system.
Rated voltage:		6 V DC --- 3 W (--- : direct current)
Exhaust:		Electromagnetic quick exhaust valve
Power supply:		4 AA batteries (R6P, LR6)
Battery duration:	Alkaline	Approx. 1000 times (170 mmHg, once/day, 73.4°F (23°C), Use Adult Cuff)
	Manganese	Approx. 300 times
Automatic Power Off function:		Approx. 3 min. (after activated)
Dimensions:		5.1 (W) x 2.1 (H) x 4.6 (D) in (130 (W) x 53 (H) x 118 (D) mm)
Weight:		Unit: Approx. 8.5 oz.(240g) w/o batteries Adult cuff: Approx. 4.6 oz.(130g)
Operating conditions:	Temperature	50°F-104°F (10°C-40°C)
	Humidity	30% to 85% RH
Storage conditions:	Temperature	-4°F-140°F (-20°C-60°C)
	Humidity	10% to 95%RH
Electric shock protection:		Internal power unit
Degree of Protection		⚡ Type B applied part
Mode of operation:		Continuous operation
Protection against ingress of water:		IPX0
Service Life:		1 year
Cuff's Life:		Approx. 2000 times
Accessories		Set includes an Adult cuff, 4 AA batteries (R6P) for the monitor, Instruction Manual, Quick Start Guide, Blood Pressure Log, FAQs.
Optional Accessories:		Large Adult Cuff 12-1/2"– 16-1/2" (32.0 – 42.0 cm)

* Applied part for this device is Cuff.

* A range in barometric pressure 105 kPa to 70 kPa

* A range in altitude -300 m to 3000 m

* Overall system accuracy of this device is met with the requirement of item 7.9 of EN1060-3.
EN1060-3 : Non-invasive sphygmomanometers Part 3 (European Standard)

1-YEAR LIMITED WARRANTY

English

Your Digital Blood Pressure CH-452 Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its Digital Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for a period of one year for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Digital Blood Pressure Monitor for commercial purposes or subjecting the Digital Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Digital Blood Pressure Monitor.

If the Digital Blood Pressure Monitor fails to conform to this limited warranty, return the Digital Blood Pressure Monitor postage prepaid along with return shipping and handling fee to: Attn: CITIZEN SYSTEMS AMERICA CORP. 363 Van Ness Way. Suite 404 Torrance, CA 90501 USA. When returning a product, please also include: a copy of your receipt, bill of sale or other comparable proof of purchase; a written description of the problem; and your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

- Eng 25 -

Appendix The use of accessories and cables other than those specified (other than CITIZEN original parts) may result in increased emissions or decreased immunity of the unit.

Guidance and manufacturer's declaration - electromagnetic emissions		
The device (CITIZEN Blood Pressure Monitor) is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	N/A	

Guidance and manufacturer's declaration - electromagnetic immunity			
The device (CITIZEN Blood Pressure Monitor) is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	N/A	N/A
Surge IEC 61000-4-5	±1 kV line to line ±2 kV line to earth	N/A	N/A
Voltage dips, short interruptions and voltage variations on power supply IEC 61000-4-11	<5 % U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycle <5% U_T (95% dip in U_T) for 5 sec.	N/A	N/A
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Note: U_T is the A.C. mains voltage prior to application of the test level.

- Eng 26 -

Guidance and manufacturer's declaration - electromagnetic immunity

The device (CITIZEN Blood Pressure Monitor) is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.

Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Recommended separation distance N/A $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^{a)} , should be less than the compliance level in each frequency range ^{b)} . Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- Field strengths from fixed transmitters, such as base stations for radio (cellular/ cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the device is used exceeds the applicable RF compliance level above, the device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the device.
- Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the device (CITIZEN Blood Pressure Monitor)

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz N/A	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	N/A	0.12	0.23
0.1	N/A	0.38	0.73
1	N/A	1.2	2.3
10	N/A	3.8	7.3
100	N/A	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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- CITIZEN is a registered trademark of Citizen Holdings Co., Japan.