
INSTRUCTION MANUAL FOR CITIZEN SELF-STORING BLOOD PRESSURE ARM MONITOR

REF **CH-456**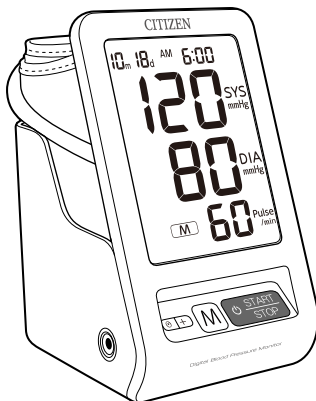

English
Español

Contents

- Ensure that you have all the following components**

- Large Adult Cuff
- AC Adapter

Blood Pressure Log

GENERAL REMARKS

General remarks on blood pressure and blood pressure measurement


1. To deflate the cuff quickly, press the "START/STOP" switch.
2. Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:
Adult Cuff fits arm circumference: 8-3/4" - 12-1/2" (22-32 cm)
[OPTION]
Large Adult Cuff fits arm circumference: 12-1/2" - 16-1/2" (32-42 cm)
3. When attaching the cuff, adjust the position of the cuff so that the blue marker is located in the center of your arm and the hem of the cuff is 0.4"-0.8"(1-2cm) for Adult Cuff/0.8"-1.2"(2-3cm) for Large Adult Cuff above your elbow.
4. The monitor is not intended to be a diagnostic device, because it is intended for home use.
5. Self measurement is not therapy! In no event should you change the dosage of your medication prescribed by your physician.
6. As preparation for measuring your blood pressure, you should urinate and then remain relatively still for 10 to 15 minutes prior to measurement.
7. Exercising, eating, drinking, smoking, etc., prior to measurement may affect the results.
8. Your blood pressure varies constantly throughout the day. Measurements should be taken regularly at the same time each day.
9. DO NOT place too much emphasis on the results of one measurement. Keep a continuing record of your blood pressure variations. A complete picture can only be obtained from a large number of readings.
10. Emotional stress tends to cause your blood pressure to rise.

To obtain correct blood pressure measurements

1. Take five or six deep breaths and then relax before measuring your blood pressure. If you are tense when taking the measurement, you will not get a valid reading.
2. Your blood pressure will be elevated if you are anxious or irritated, suffering from lack of sleep or constipation, or have just taken some exercise or eaten a meal.
3. Wrap the cuff around your arm and tighten it so that one finger can be inserted between the cuff and arm. ( See page 16)
4. DO NOT measure your blood pressure after bathing or drinking.
5. If you feel the need to urinate, do so before measuring your blood pressure.

GENERAL REMARKS

6. Measure your blood pressure where the room temperature is around 68°F (20°C). DO NOT measure your blood pressure when it is too cold (below 50°F (10°C)) or too hot (above 104°F (40°C)) in the room.
7. DO NOT try to measure your blood pressure immediately after drinking coffee or tea or after smoking.
8. Measure your blood pressure when you are relaxed and still. Keep the center of the cuff at the level of your heart and DO NOT move your arm or talk.
9. DO NOT measure blood pressure when the cuff has been on your arm for a few minutes or more. During this time, your arm will have built up a higher blood pressure and you will not get valid reading.
10. If the Body Movement Indicator or Irregular Heartbeat (IHB) Indicator is displayed, relax and measure your blood pressure again. If the Irregular Heartbeat (IHB) Indicator appears frequently, you are advised to consult a doctor about your health condition. ( See page 19)
11. The figure below shows the blood pressure classifications under the WHO standards.

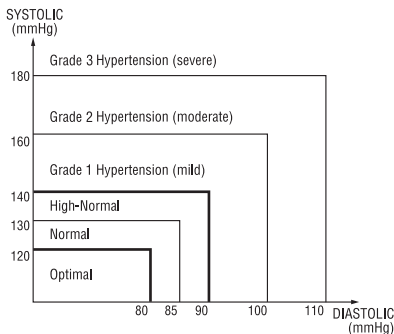Remarks:

The graph is not exact, but may be used as a guide in understanding non-invasive blood pressure measurements. The device is only intended for use by adults.

Definitions and Classifications of Blood Pressure Levels

Measure your blood pressure at the same time each day.

- * Your blood pressure changes all the time. This means that data gathered over a long period has far more significance than data from just one measurement. For this reason, you must measure your blood pressure on a daily basis. Ideally, you should measure your blood pressure at the same time each day, wherever possible.

PRECAUTIONS FOR USE AND MAINTENANCE

Precautions for use

1. If you suffer from heart disease, high blood pressure or other circulatory disease, consult your physician before using the monitor.
2. Inflating to a higher pressure may result in bruising where the cuff is applied. If the cuff pressure feels abnormal or you experience any other irregularity while using the cuff, reduce the pressure immediately by pressing the "START/STOP" switch, detach the cuff or unplug the air hose connector from the monitor and then consult the sales outlet where you purchased the monitor.
3. If you think the measurement is abnormal or if measurement makes you feel unwell, discontinue use and consult your physician.
4. Blood pressure measurement may not be possible for anyone with a weak pulse or arrhythmia.
5. Repeated blood pressure measurement and prolonged overinflation may cause problems such as congestion or swelling in some people.
6. Frequently repeated blood pressure measurements will not give accurate results. Allow an interval of about 1 minute between measurements.
7. If you suffer from a severe problem with blood circulation in your arms, consult your physician before using the monitor. Failure to do so could be hazardous to your health.
8. Measurement may not be possible for anyone with insufficient blood flow to the area where measurements will be taken or who suffers from a frequent irregular heartbeat. Consult your physician for advice on whether to use the monitor.
9. DO NOT wrap the cuff around an injured arm.
10. DO NOT wrap the cuff around an arm in which a drip (intravenous infusion) is inserted or which is being used for blood transfusion as part of medical treatment. Doing so could result in an injury or a serious accident.
11. DO NOT use the monitor in the vicinity of flammable gases such as those used for anaesthesia. Doing so could ignite the gases and cause an explosion.
12. DO NOT use the monitor in enriched oxygen environments such as a hospital's hyperbaric chamber or oxygen tent. Doing so could ignite the oxygen and cause a fire.
13. DO NOT use mobile phones near the monitor as this could result in a malfunction.
14. If you use a cardiac pacemaker, consult your physician before using the monitor.
15. Be sure to use this unit only for measuring blood pressure. DO NOT use it for any other purpose.

PRECAUTIONS FOR USE AND MAINTENANCE

16. Be sure to use only our authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.
17. DO NOT use this unit on infants.
18. Blood pressure measurement may not be possible for anyone with common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation.
19. This product is designed for use over an extended period of time; however, it is generally recommended that it be inspected every two years to ensure proper function and performance.

Maintenance Precautions

1. DO NOT store the blood pressure monitor in locations exposed to direct sunlight, high temperatures (over 140°F (60°C)), low temperatures (below -4°F (-20°C)), high relative humidity (over 95%) or excessive amounts of dust.
2. DO NOT drop the blood pressure monitor or the cuff and subject it to other shocks or vibration.
3. Remove the batteries if the monitor will be left unused for a long period.
4. DO NOT attempt to disassemble the monitor or the cuff.
5. DO NOT bend the cuff or the air hose excessively.
6. If the monitor is very dirty, wipe it clean with a cloth moistened with sterilizing alcohol or a neutral detergent. Then wipe it with a dry cloth.
7. NEVER clean the blood pressure monitor with thinners or benzene, as this could damage the monitor.
8. To clean the cuff, wipe it with a moist cloth. Avoid hard rubbing, as this will cause air leakages. Take care also not to get water into the air hose.

Warning

The device has been tested and homologated in accordance with IEC60601-1-2 for EMC. This does not guarantee in any way that the device will not be affected by electromagnetic interference. Avoid using the device in high electromagnetic environment.

Toll-Free Customer Care Help Line
866-326-1313 Mon-Fri 8:30 am – 4:30 pm CST

Distributed in USA by:
Veridian Healthcare, LLC
1465 S. Lakeside Drive
Waukegan, IL 60085 USA
Made in China

#93-1097 05/11

FCC STATEMENT

NOTE:

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This digital apparatus has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The digital apparatus generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the digital apparatus does cause harmful interference to radio or television reception, which can be determined by turning the digital apparatus on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the digital apparatus and the receiver.
- Connect the digital apparatus into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)

This Class B* digital apparatus complies with Canadian ICES-003.

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
Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

IDENTIFICATION OF PARTS

Component names

<Main Unit>


English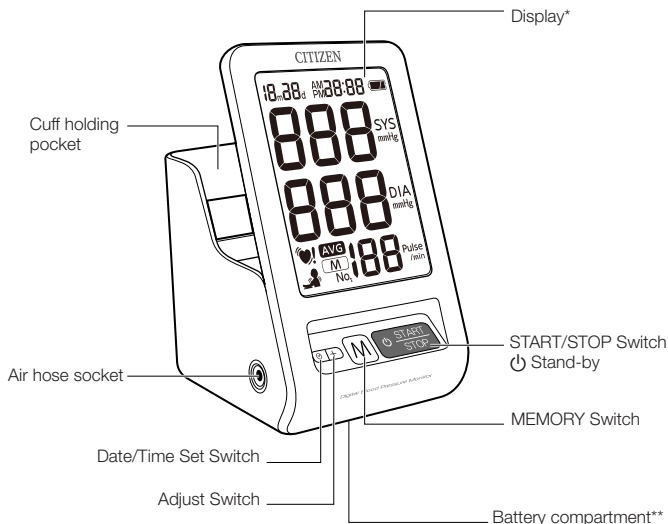


*See page 9 for the display icons.

**See page 10 for the Battery compartment

<Cuff>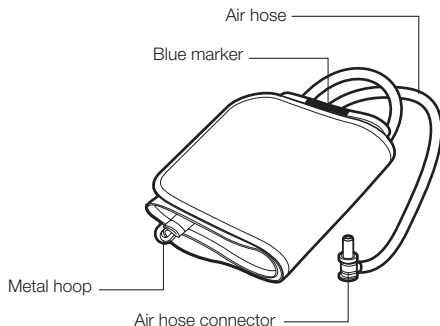

<Display>

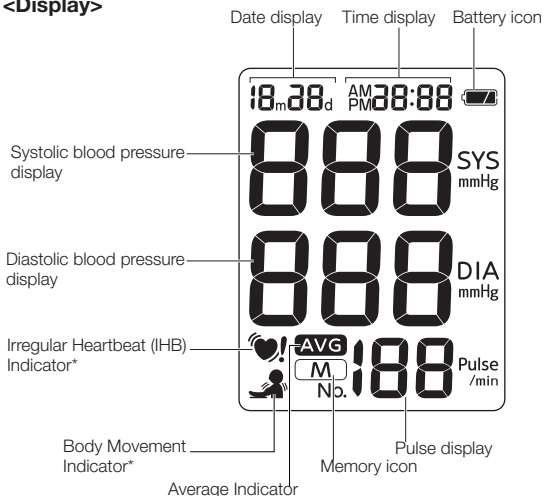* See page 19 for the Body Movement Indicator and Irregular Heartbeat (IHB) Indicator.


LOADING THE BATTERIES

(Batteries supplied with the unit should be loaded in compartment before you use the blood pressure monitor.)

- * When you install the batteries for the first time after purchase, the monitor will display the clock setting screen. Follow step 2 and subsequent steps in TIME ADJUSTMENT ( See page 13) to set the clock.

1 Open the cover of battery compartment.

Pressing the hook down, pull the cover toward you to open.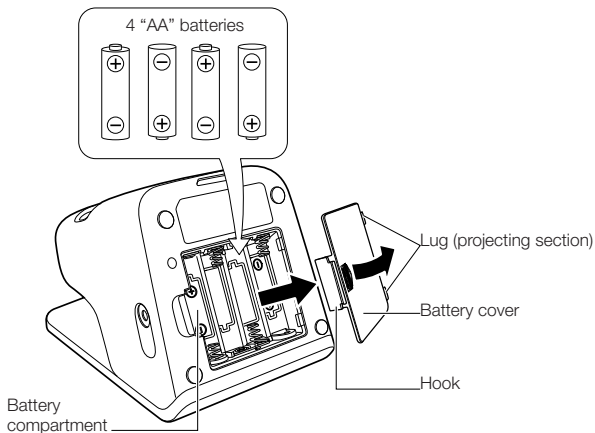


- * If the Blood pressure monitor is left unused for an extended period of time, the batteries should be removed from the compartment.

2 Place batteries in the compartment.

Put the batteries paying attention to the positive and negative terminal symbols $\oplus$ and $\ominus$.

3 Close the battery cover.

Fitting the lugs of the cover at the mating recesses of the battery compartment body, push the hook to close the cover.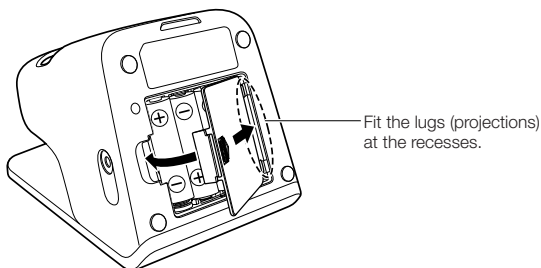


REMOVING THE BATTERIES

- When the icon  is displayed or nothing appears in the display, replace the old batteries with four new ones. All four batteries should be changed at once.
- : The remaining capacity of the batteries is inadequate.
- Adjust the clock after changing the batteries.
- The data stored in the memory is not deleted by changing the batteries.
- Do not use rechargeable batteries.
- Do not use alkaline (LR6) and manganese (R6P) batteries together.
- Batteries included are for demonstration purposes only. Battery life may be shorter than specified.
- When disposing of used batteries, comply with governmental regulations or environmental public institution's rules that apply in your country/area.

USING AC ADAPTER (OPTION)

Using the optional dedicated AC adapter (model: AC-230CZ) for the CITIZEN blood pressure monitor (CH-456) allows you to measure your blood pressure without having to worry about the amount of remaining charge in the batteries.)

- 1 Insert the AC adapter plug into the socket on the blood pressure monitor.**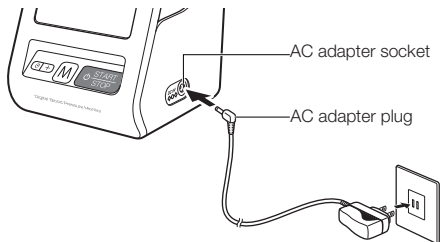

- 2 Plug the AC adapter into a mains power outlet.**

- If you plan to use the AC adapter for an extended period of time, remove the batteries. If the batteries are left in the compartment, electrolyte may leak from the batteries and cause a fault in the blood pressure monitor.
- When only batteries are used to supply power to the monitor, the AC adapter should be unplugged from the monitor. Leaving the AC adapter plugged into the blood pressure monitor could cause a malfunction.

The dedicated AC adapter (model: AC-230CZ) for the CH-456 is optional. The AC adapter is available from retailers stocking CITIZEN digital blood pressure monitors.

TIME ADJUSTMENT


Setting the clock enables measurements to be accurately recorded.

The monitor can record the date and time of measurement as well as the measurement results. Be sure to set the clock after inserting the batteries. Set the date first and then the time.

- * When you install the batteries for the first time, the monitor will display the clock setting screen. You do not need to perform step 1. Proceed to step 2. If you set the clock and change its setting, start from step 1.

1 Press and hold the Date/Time Set switch for approximately two seconds.

The "year" indicator blinks.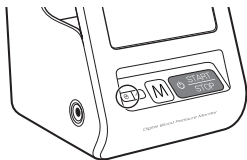

2 Press the Adjust switch to adjust the year.

Press  Adjust switch to increase the number by one. (The year can be adjusted in the range of 2011-2040.)

Press  Date/Time Set switch to confirm the setting.

The "year" is set and the "month" indicator blinks.

- * You can fast-forward the numbers in display by pressing and holding  Adjust switch.


3 Press the Adjust switch to adjust the "month".

Press  Adjust switch to increase the number by one.

Press  Date/Time Set switch to confirm the setting.

The "month" is set and the "day" indicator blinks.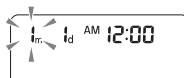


4 Press the Adjust switch to adjust the "day".

Press  Adjust switch to increase the number by one.

Press  Date/Time Set switch to confirm the setting.

The "day" is set and the "hour" indicator blinks.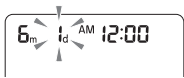


TIME ADJUSTMENT

5 Press the Adjust switch to adjust the "hour".

Press  Adjust switch to increase the number by one.

Press  Date/Time Set switch to confirm the setting.

The "hour" is set and the "minutes" indicator blinks.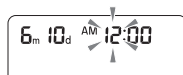

6 Press the Adjust switch to adjust the "minutes".

Press  Adjust switch to increase the number by one.

Press  Date/Time Set switch to confirm the setting.

The "minutes" are set and "12" (12-hour format) blinks.


7 Press the Adjust switch to adjust the "time format".

The setting switches between 12-hour format (12H) and 24-hour format (24H) each time  Adjust switch is pressed.

Press  Date/Time Set switch to confirm the setting (12H or 24H).

When clock adjustment is completed, the set date and time appear briefly and then the display turns off.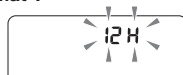

* The "minute" count (from 0 seconds) starts when  Date/Time Set switch is pressed.

The accurate time can be set by pressing  Date/Time Set switch.

MEASURING YOUR BLOOD PRESSURE

1 ATTACHING THE CUFF

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:

Adult Cuff fits arm circumference: 8-3/4" - 12-1/2" (22-32 cm)


[OPTION]

Large Adult Cuff fits arm circumference: 12-1/2" - 16-1/2" (32-42 cm)


- 1 Unroll the cuff and thread the end through the metal hoop so that the side with the hook-and-loop fastener is on the outside.

- 2 Put the cuff around your arm so that the blue marker comes to the palm side.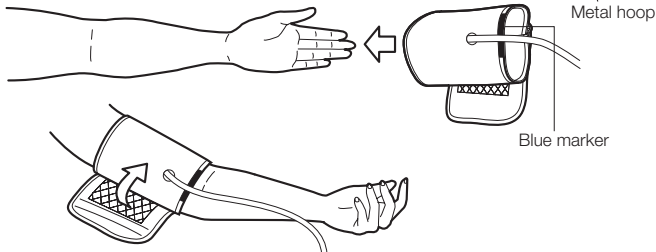

- 3 Adjust the position of the cuff so that the blue marker is located in the center of your arm and the hem of the cuff is 0.4"-0.8"(1-2cm) for Adult Cuff/0.8"-1.2"(2-3cm) for Large Adult Cuff above your elbow.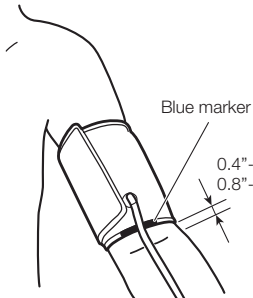

* The cuff should be put on the bare arm or over a light-weight underwear.


* If you wear a heavy-weight top, please remove it.

0.4"-0.8"(1-2cm) for Adult Cuff


0.8"-1.2"(2-3cm) for Large Adult Cuff

MEASURING YOUR BLOOD PRESSURE

- 4** Pull the end of the cuff outwards so that the cuff is snug around your arm and then secure the end to the hook-and-loop fastener.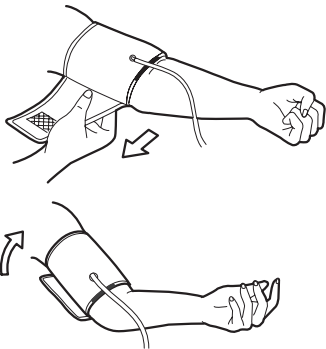

- 5** Place your arm on a table or the like, so that the center of the cuff comes to the height of your heart.

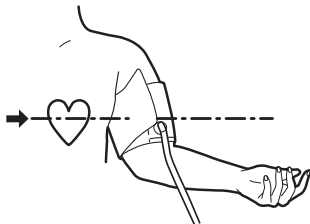
* Carefully place the cuff on your arm, taking care to use the correct tightness. The tightness is correct if you can readily slide a finger between the cuff and your arm.

- 6** Relax your arm and lightly open your hand, with the palm upward.

2 Plug the air hose connector into the main unit.

- Plug the air hose connector firmly into the air hose socket on the main unit, as illustrated.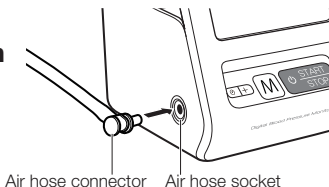


3 Place the cuff at the height of the heart.

(See page 20)

4 Press the START/STOP switch (the blue switch).

- All digits displayed is an initial display, indicating that the monitor is functioning normal.
- The cuff is pressurized automatically.

* If the unit judges that pressurization is insufficient, it automatically repressurizes.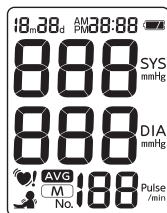


* ♥ is displayed when a pulse is detected.

* When you want to stop measurement, press the "START/STOP" switch. The cuff is deflated and measurement stops.

* If the pressurization value exceeds 280mmHg, if pressurization feels abnormal, or if you want to stop the measurement process, press the START/STOP switch again. The cuff deflates, and the power turns off.

5 The measurement results are displayed.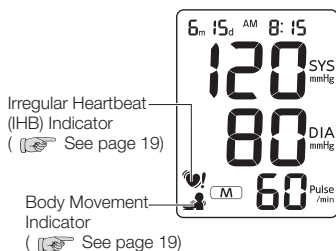


- Once measurement is complete, the cuff deflates and the measurement results (systolic/diastolic blood pressure values, pulse) are displayed.

* Provided there is no error in the measurement results, they are stored automatically.


6 Finishing measurement.

- Press the "START/STOP" switch to turn the monitor off.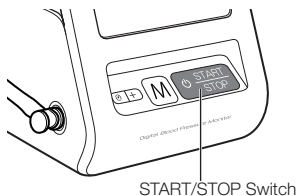

[Manual pressurization]

- If your systolic blood pressure is expected to exceed 200mmHg, use Manual Pressurization when taking a measurement.
- Keep "START/STOP" switch held pressed until the pressure value you want to stop pressurization is reached (approximately 40mmHg higher than the Systolic blood pressure). You can stop pressurization by the unit by releasing the switch at the aforementioned value is reached. The upper pressure limit is 280mmHg

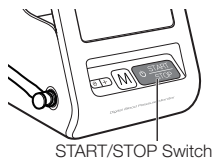The Body Movement Indicator is displayed on the measurement results display if you move your hand or arm during measurement, causing a substantial pressure change to be detected.

- If the Body Movement Indicator is displayed, measure your blood pressure again.

If an irregular heartbeat is detected during measurement, the Irregular Heartbeat (IHB) Indicator is displayed after measurement is completed.

An irregular heart beat is defined as a heart beat that varies by $\pm 25\%$ from the middle of 5 heart beat intervals during the blood pressure measurement.

- Measurement may not be correct if your heartbeat fluctuates greatly during measurement. If the Irregular Heartbeat (IHB) Indicator is displayed, measure your blood pressure again.

Caution

If the Irregular Heartbeat (IHB) Indicator appears frequently, you are advised to consult a doctor about your health condition.


AUTOMATIC POWER OFF FEATURE

- After taking the blood pressure measurement, pressing the “START/STOP” switch will turn the monitor off. However, if you do not press the switch, the monitor’s Auto OFF function will automatically turn the monitor off after 3 minutes.

TIPS FOR OBTAINING ACCURATE MEASUREMENTS


Your blood pressure varies according to your posture, the time of day and a range of other factors. Ideally, you should measure your blood pressure in the same posture at the same time every day.

■ Taking measurements while sitting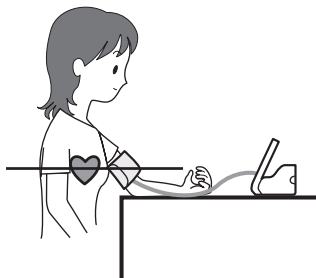

- 1 Place your arm on a table or similar surface with your forearm extended.
- 2 It is important to have the cuff level with your heart.
- 3 Open your hand slightly on the table so that your palm is facing up and your fingers are relaxed.
- 4 Do not move your body or talk while taking the measurement.

■ Measurement in a reclining posture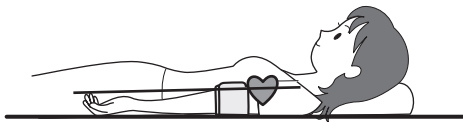

* Relax yourself to avoid placing pressure on the cuff.

- 1 Lie down, facing the ceiling.
- 2 Lightly open your hand with palm up and straighten your arm.
- 3 Relax your body, arm and fingers.
- 4 Do not move your body or talk during measurement.

■ You may use your right arm to measure your blood pressure.


Blood pressure value is likely to differ by as much as 10mmHg when measured on the right arm instead of the left (or vice-versa). Measure your blood pressure on the same arm each day.

CALLING UP THE DATA STORED IN MEMORY


90 measurement results can be stored in Memory.

The average value is calculated automatically to help you manage your daily health.

- 1 Press the **M** Memory switch once.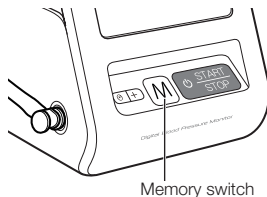

* Example of display when the **M** Memory switch is pressed.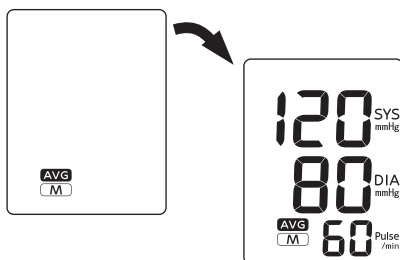


AVG indicating the average value appears at the bottom of the display. Then the average value, based on the last three measurements, is displayed.

* (**AVG** and the average value do not appear if there are two or fewer measurements stored in memory.)

Each time you press the switch, the measurement data are displayed in order from the most recent to the oldest.

* You can fast-forward the data numbers in display by pressing and holding the  Memory switch.

* Example of display when the  Memory switch is pressed.

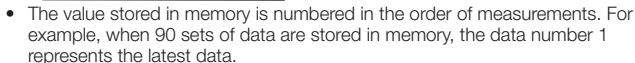Data number: 1, 2, 3, 89, 90

The latest data The earliest data

- 1 While pressing the Memory switch, press and hold the “START/STOP” switch for more than 3 seconds.

All data stored in Memory will be deleted.

Deleted

A message saying all the data have been deleted from the memory is displayed.


KEY TO DISPLAY ICONS

DISPLAY MARK	CONDITION/CAUSE	CORRECTIVE ACTION
	Appears when the cuff is attached too loose.	Attach the cuff properly and measure again. ( See page 15 "ATTACHING THE CUFF")
	Appears when the cuff is not attached correctly and the pulse cannot be detected.	
	Appears when the pressure is 300mmHg or higher.	
 or 	Appears when the batteries have run out.	Replace the batteries. ( See page 11 "REMOVING THE BATTERIES".)
	Appears when the pulse rate falls outside the measuring range (below 39 beats/min. or above 181 beats/min.)	Attach the cuff properly again. Take deep breaths to relax, and measure again. ( See page 20 "TIPS FOR OBTAINING ACCURATE MEASUREMENT".)
 	The unit does not function properly.	Contact the store where it was purchased or your local service center. ( See page 26 "BEFORE REQUESTING REPAIRS OR TESTING".)

ABOUT “BLOOD PRESSURE”

What is blood pressure?


The heart is a pump that circulates blood throughout your body. Blood is pumped from the heart at a constant pressure into arteries. This pressure is called the arterial blood pressure and represents, in general terms, your blood pressure. Blood pressure is indicated by several kinds of pressures, including the systolic pressure that occurs when the heart pumps blood, and the diastolic pressure that occurs when blood returns to the heart.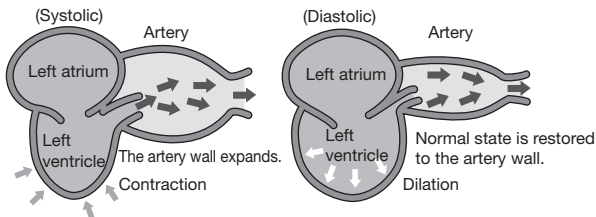

Your blood pressure changes all the time.

Your blood pressure differs according to your age, gender and a range of other factors. It is likely to be affected by your biorhythm during the day and by your posture, physical activities, mental activities, level of stress and even by the air temperature. The blood pressure of a healthy person generally varies within a day.

[Example of blood pressure variations within a day]

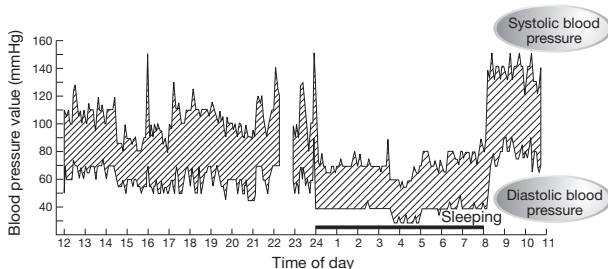(Blood pressure values measured at five-minute intervals through the day)

● Bevan AT, Honour AJ, Stott FH. Clin Sci 1969;36:329–44.

BLOOD PRESSURE Q&A

- Q** Why is the blood pressure measured at home different from that measured by the physician or at a hospital?
- A** Your blood pressure may vary due to exercise, the ambient temperature or your mental state. When you have a physician or nurse measure your blood pressure, the pressure value is likely to be 10 to 20 mmHg higher than usual due to anxiety and/or stress. Knowing what affects your blood pressure helps you to better monitor your health.
- Q** Why does the blood pressure obtained vary with measurement?
- A** Our blood pressure is adjusted by automatic nerve function. Blood pressure differs with every heart beat. We are likely to think that our blood pressure is constant, but it varies if you take measurements in repetition. Blood pressure is susceptible to the time of day, such as morning and afternoon, season and atmospheric temperature. In addition, blood pressure is easily affected by mental stress or emotional ups and downs. It tends to increase when you feel tension or decrease when you are relaxed.
- Q** What is the benefit of measuring my blood pressure at home?
- A** Blood pressure measurements taken at home give reliable data as they can be taken when you are in a stable and relaxed condition. Physicians place considerable importance on blood pressure measurements taken at home. You should measure your blood pressure at the same time each day and record the variations in your blood pressure.

Always consult your physician for an interpretation of your blood pressure readings and to determine the proper treatment.

BEFORE REQUESTING REPAIRS OR TESTING

Make sure to check the following before sending your blood pressure monitor away to be tested or repaired.

English

Problem	Checks	Response
Display stays blank when you press the "START/STOP" switch.	Check whether the batteries are dead.	Replace all the batteries with new ones.
	Check whether the batteries are installed the right way round ($\oplus$ and $\ominus$ oriented correctly).	Load the batteries in the correct direction.
No measurements can be taken.	Check whether the measurement icon  is displayed.	Attach the cuff properly.
	Check whether the cuff is attached correctly.	
	Did you stay calm during measurement?	Measure your blood pressure again, making sure to remain still.
	Note that blood pressure measurement may not be possible for someone with an extremely weak pulse or cardiac arrhythmia (irregular pulse).	
Your blood pressure readings are abnormally high or low compared with those taken in hospital.	 See page 25.	
Other phenomena	Remove the batteries from the unit and change them with new ones.	

* If you want to dispose this product, do not mix with general household waste. There is a separate collection systems for used electronics products in accordance with legislation.

SPECIFICATIONS

Model Number:		CH-456	
Measurement system:		Oscillometric method	
Measurement Localization		Upper Arm	
Cuff:		Soft cuff	
Adult cuff circumference range:		8-3/4" - 12-1/2" (22.0-32.0 cm)	
Measurement range:	Pressure	0 to 280 mmHg	
	Pulse	40 to 180 pulse/min	
Accuracy:	Pressure	±3 mmHg	
	Pulse	±5% of reading	
LCD displays:	Pressure	3 digits	
	Pulse	3 digits	
	Icons	♥: Measurement icon	
		🔋: Battery icon	
		💓: Irregular Heartbeat (IHB) Indicator	
		🚶: Body Movement Indicator	
		📌: Memory icon	
		AVG: Average indicator	
Switch:		4 (⏻, +, START/STOP, MEMORY)	
Inflation:		Automatic Inflation by internal pump	
Deflation:		Automatic speed deflation system.	
Rated voltage:		6V DC --- 3W (--- : direct current)	
Exhaust:		Electromagnetic quick exhaust valve	
Power supply:		4 AA batteries (R6P, LR6)	
Battery duration:	Alkaline	Approx. 500 times	(170 mmHg, once/day, 73.4°F (23°C), Use Adult Cuff)
	Manganese	Approx. 150 times	
Automatic Power Off function:		Approx. 3 min. (after activated)	
Main Unit Dimensions:		4.1 (W) X 6.5 (H) X 4.8 (D) in (105 (W) X 166 (H) X 122 (D) mm)	
Cuff Size:	Adult Cuff	Approx. 5.7 (W) x 19.4 (L) in (144 x 492 mm)	
Weight:		Unit: Approx. 13.8 oz.(390g) w/o batteries	
		Adult cuff: Approx. 4.6 oz.(130g)	
Operating conditions:	Temperature	50°F-104°F (10°C-40°C)	
	Humidity	30% to 85% RH	
Storage conditions:	Temperature	-4°F-140°F (-20°C-60°C)	
	Humidity	10% to 95%RH	
Electric shock protection:		Internal power unit	
Safety Classification:		⚡ Type B Equipment	
Mode of operation:		Continuous operation	
Memory:		90 readings, Average of last 3 readings	
Accessories:		Set includes an Adult cuff, 4 AA batteries (R6P) for the monitor, Instruction Manual, Quick Start Guide, Blood Pressure Log, FAQ's.	
Optional Accessories:		Large Adult Cuff 12-1/2"– 16-1/2" (32.0 – 42.0 cm), AC Adapter	

- Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the limits prescribed by the American National Standard, *Manual, electronic, or automated sphygmomanometers*.

: Refer to instruction manual before use

5-YEAR LIMITED WARRANTY

Congratulations on your purchase of a Digital Blood Pressure Monitor. Your Digital Blood Pressure Monitor is covered by the following limited warranty commencing upon the date of purchase, and subject to the following terms and conditions:

The Warrantor warrants that its Digital Blood Pressure Monitor will be free from defects in materials and workmanship under normal consumer usage for a period of five years for the original purchaser of the product.

Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the Digital Blood Pressure Monitor for commercial purposes or subjecting the Digital Blood Pressure Monitor to abnormal usage or conditions; or (d) other acts which are not the fault of the Warrantor, are excluded from coverage. This warranty does not cover batteries or other power sources that may be provided with, or used with the Digital Blood Pressure Monitor.

If the Digital Blood Pressure Monitor fails to conform to this limited warranty, return the Digital Blood Pressure Monitor postage prepaid along with \$5.00 for return shipping, handling and insurance to: Attn: Repair Department, 1465 S. Lakeside Drive, Waukegan, IL, 60085. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the Warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

Distributed in USA by:
Veridian Healthcare, LLC
1465 S. Lakeside Drive
Waukegan, IL 60085 USA
Made in China

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- CITIZEN is a registered trade mark of Citizen Holdings Co., Ltd., Japan.
CITIZEN es una marca registrada de Citizen Holdings Co., Ltd., Japón.
CITIZEN SYSTEMS JAPAN CO., LTD.