
## English Manual COMPRESSOR NEBULIZER CNC701

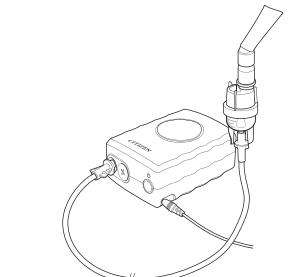
### Instruction Manual

#### INTENDED USE


The CNC701 Compressor Nebulizer is intended to be used for inhaling medication for respiratory disorders.

#### PRODUCT DESCRIPTION AND CONTENTS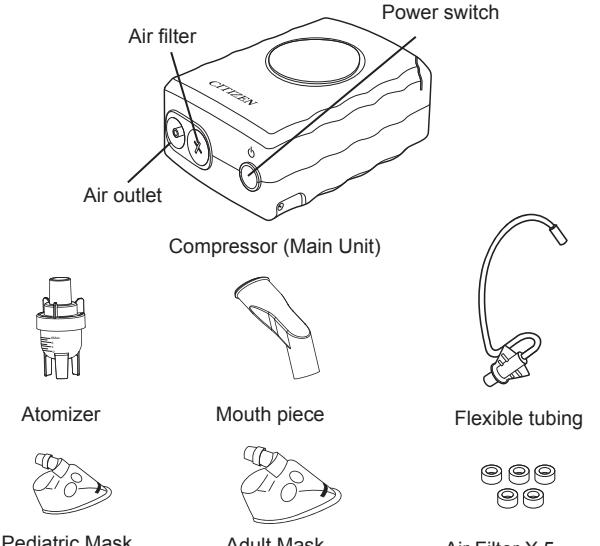

#### WARNINGS

- Always consult your medical practitioner before using this appliance.
- Should you feel any discomfort during use, immediately discontinue use of the device and consult your doctor or physician.
- Do not use water alone in the nebulizer for inhaling purposes.
- Never cover the compressor with a blanket, towel, or any other covering during use.
- Avoid using in locations where the device may be exposed to flammable gases.
- Avoid using or storing in locations where the device may be exposed to noxious fumes or volatile substances.
- Do not use the device in any anesthetic or ventilator breathing circuits.
- Always unplug the appliance from the power supply at the end of every treatment session.
- Never operate the device with a damaged power cord or plug.
- Keep the nebulizer out of reach of children.
- Store the parts in a clean location to avoid contamination.
- Make sure the power cord is not in contact or near any heated surfaces.
- Never use the appliance continuously for longer than indicated on the label attached to the bottom of the appliance.

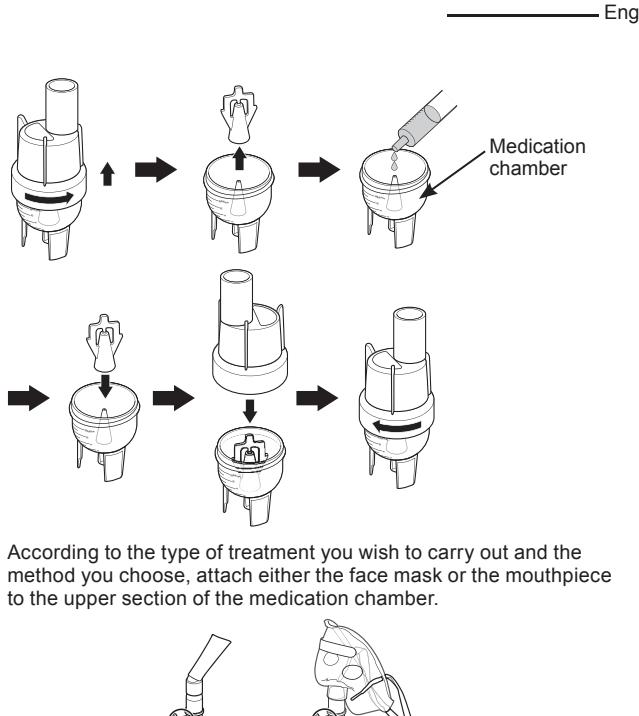Your appliance is now ready to use.

#### To carry out aerosol-therapy treatment

- Plug your appliance into a socket of the correct voltage.
- Press the switch of the appliance with
- Start the treatment and carry on for as long as you can still see the 'mist' produced by nebulization of the medication performed by the appliance.
- One treatment should be used for less than 45min (equipped with 45min-timer) and after the treatment should be taken a break at least 45 min off.

**Warning:** the presence of a small amount (about 0.4 ml) of medication in the medication chamber at the end of each treatment is absolutely normal. This amount, called residue volume, cannot be nebulized.

#### At the end of each treatment

- Turn off the appliance by pressing its switch with
- Unplug the appliance from the power socket.
- Wind up the power supply cord.
- Detach the face mask (or mouthpiece) from the medication chamber.
- Detach the flexible tubing from both the medication chamber and the appliance's air outlet.
- Wipe the face mask (or mouthpiece) and the medication chamber, clean according to the instructions in the following paragraph.

#### CLEANING

To avoid inefficient nebulization or contamination, clean all parts after each use, making sure to remove any residual medication.

#### ■ Washable parts

Wash with warm water and mild detergent (neutral detergent). Rinse each part thoroughly with clean, hot tap water and allow them to air dry in a clean location.

#### ■ Non-washable parts

Compressor, Atomizer  
After making sure that the power plug is unplugged from the power outlet, wipe clean with a soft cloth moistened with water or a mild detergent (neutral detergent).

Air Filter  
Do not wash or clean the filter. If the filter becomes damp or wet for any reason, replace it immediately as it may cause air blockage.

**DISINFECTION**  
Desinfect the parts once a week. If the parts are heavily stained, replace them with new ones.

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