



INSTRUCTION MANUAL FOR COMPRESSOR NEBULIZER

GC806

2001

FIG. A

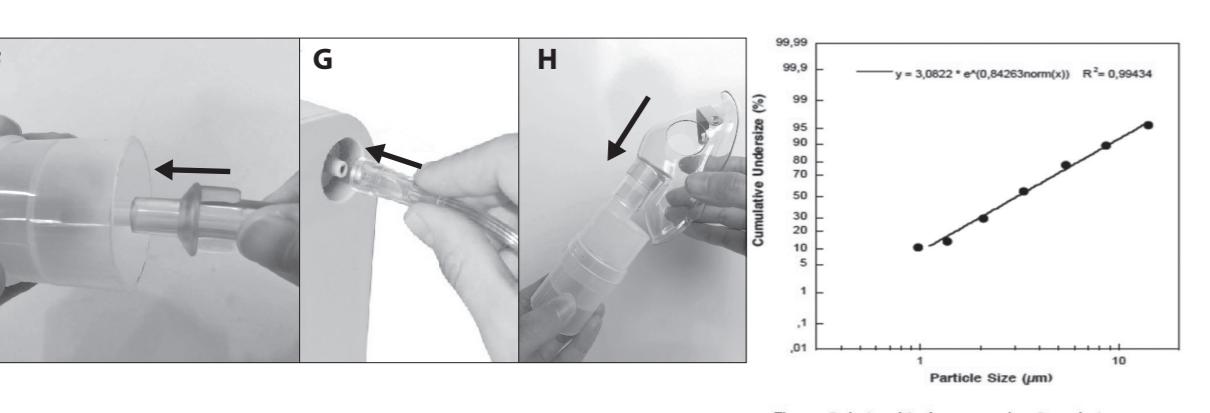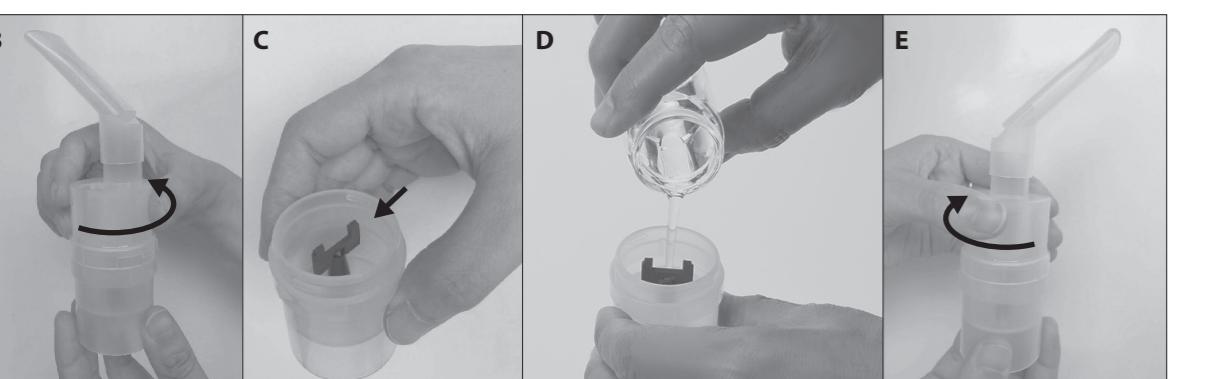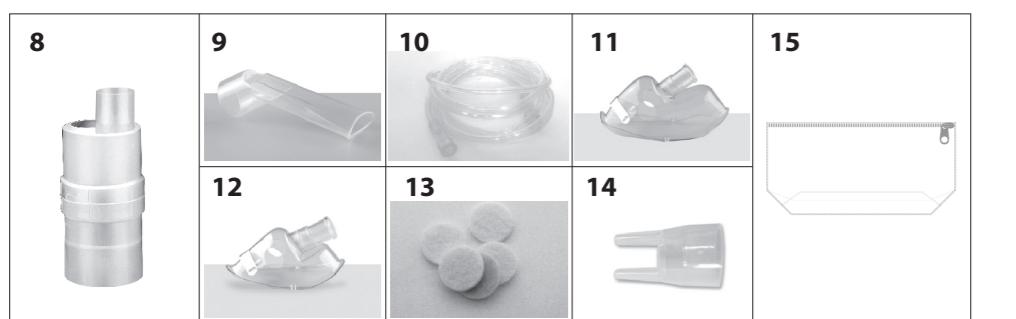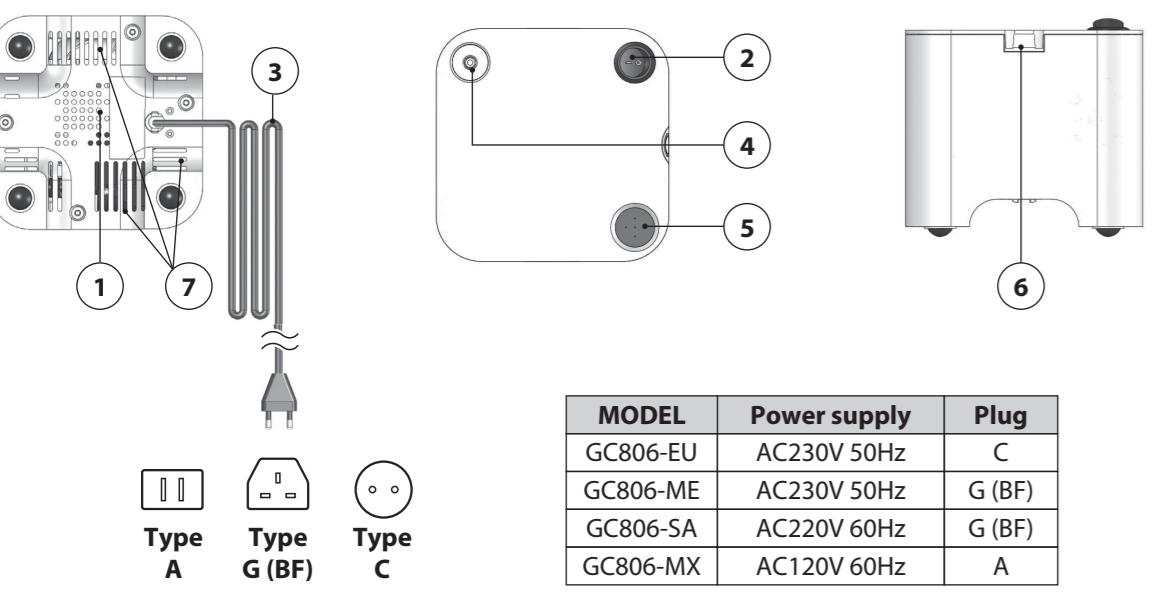EC REP EC- Representative:
Donawa Lifescience Consulting Srl
Piazza Albania, 10
00153 Rome / Italy

Manufacturer:
Globalcare Medical Technology Co., Ltd.
No. 7 factory building, European Industrial Zone,
No. 39 Middle Industrial Main Road, Xiaolan Town, Zhongshan City

CITIZEN is a registered trademark of Citizen Watch Co., Ltd. Japan.
Design and specifications are subject to change without notice.

CITIZEN SYSTEMS JAPAN CO., LTD.
6-12, Tanashi-cho, Nishi-Tokyo- shi,
Tokyo 188-8511, Japan
E-mail: sales-eo@systems.citizen.co.jp
http://www.citizen-systems.co.jp/

English	GC806 COMPONENTS - (PIC. A)
1. Compressor housing	6. Nebulizer holder
2. ON/OFF switch	7. Nebulizer tube openings
3. Power cord	8. Nebulizer
4. Air outlet	9. Mouthpiece
5. Filter	10. Air tube
	11. Adult mask
	12. Spare air filters
	13. Spare air filters
	14. Nosopiece
	15. Accessories case

IMPORTANT: READ CAREFULLY THE INSTRUCTIONS FOR USE BEFORE OPERATION.**GENERAL INFORMATION**

GC806 is an aerosol therapy system suitable for domestic use.
1. Use the device only as described in this manual and therefore as an aerosolotherapy system, following the indications of your doctor. Any use different from the intended one is to be considered improper and hence dangerous; the manufacturer cannot be held liable for any damage caused by improper, incorrect or unreasonable use, or if the equipment is connected to electrical installations which do not comply with current safety regulations.
2. Keep this manual for future reference.
3. Do not operate the unit in presence of any anaesthetic mixture inflammable with oxygen or nitrogen protoxide.
4. The correct functioning of the equipment can be affected by electromagnetic interferences which exceed the limits indicated by the European standards in force. In case this device interferes with other electrical devices, move it and plug it to a different power socket.
5. In case of failure or malfunction, read the "POSSIBLE PROBLEMS AND HOW TO SOLVE THEM" section. Do not tamper or open the compressor housing.
6. Use the dedicated spare parts only.
7. Comply with the safety regulations concerning the electrical devices and in particular:

- use only original accessories and components;
- never touch the unit with its hands;
- never touch the unit if it is not protected against splashes;
- never touch the unit with wet or moist hands;
- do not leave the unit exposed to the weather elements;
- place the unit on a stable and horizontal surface during its operation;
- if the product is dropped by your foot or hand accidentally catching the power cord, it may cause damage or injury. Keep the product in a stable place and be careful when moving it;
- the use of this device by children or persons with full mental faculties is prohibited. If the user does not respect these rules always wears close supervision by an adult with full mental faculties;
- never touch the unit with its hands;
- do not plug the air vent for the compressor housing;
- do not block the air vent for the compressor housing;

8. Before plugging in the device, make sure that the electrical rating, shown on the rating plate on the bottom of the unit, corresponds to the mains rating.
9. In case the power plug provided with the device does not fit your wall socket, address to qualified personnel for the replacement of the plug with a suitable one. In general, the use of adapters, simple or multiple, or/extension cable is not recommended. If their use is indispensable, it is necessary to use that complying with safety regulations, paying however attention that they do not exceed the maximum power rating of the device.
10. Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not operated.
11. The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or things, for which the manufacturer cannot be held responsible.
12. The power cord of this device cannot be replaced by the user. In case of a power cord damage, address to a technical service centre authorized by the manufacturer for its replacement.
13. Always use the device in a well-ventilated environment, avoiding risks of suffocation.
14. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
15. Some parts of the unit are so small that they may be swallowed by children; keep the equipment out from children's reach.
16. If you decide not to use the device any longer, it is recommended to dispose of it according to the current regulations.
17. Make sure to:

- use this device only with medicines prescribed by your doctor;
- make the treatment under the supervision recommended by your doctor depending on the pathology;
- never touch the unit with its hands;
- only bring them as close as possible;

18. Check in the medicine package leaflet for possible contraindications for use with common aerosolotherapy systems.
19. To avoid strangulation and entanglement, keep cable and air tubes out of reach of young children.
20. Do not position the equipment so that it is difficult to operate the disconnection device.
21. For greater hygiene safety, we recommend you to avoid using the same accessories for more than one person.
22. Insert the air tube firmly into the air outlet. The air tube may come off when using the product.
23. If you use it continuously for a long period of time, the compressor housing may heat up.
INSPECTION FOR USE

- The device must be checked before each use, in order to detect possible functioning anomalies and/or damages due to transport and/or storage.
- During inhalation, sit upright and relaxed at a table and not in an armchair, in order to avoid compressing your respiratory airways and impairing the treatment effectiveness.
- The accessories must be used only with a single patient; it is not recommended to use them with several patients.
- 1. After unpacking the device, check it for visible damages or defects; pay particular attention to cracks in the plastic housing, which may appear during transport.
- 2. Before using the device, proceed with the cleaning operations as described in the 'CLEANING AND MAINTENANCE' section.
- 3. Open the nebulizer by turning counterclockwise the top (Pic. B).
- 4. Make sure that the medicine conduct cone is properly fitted on the air conduction cone inside the nebulizer (Pic. C).
- 5. Put the prescribed quantity of medicine into the nebulizer (Pic. D).
- 6. Close the nebulizer by turning clockwise the two parts, paying attention that they are thoroughly sealed (Pic. E).
- 7. Connect one end of the I/O to the nebulizer and the other end to the air outlet (Pic. F) on the device.
- 8. If you need to clean the tube, insert it directly onto the nebulizer (Pic. H).
- 9. Plug the device into the wall socket, making sure that the main power corresponds to the electrical rating of the device.
- 10. To start the treatment, set the I/O switch into the **I** position.

ATTENTION: This device is for intermittent use: 20 min ON / 40 min OFF. * Do not use the product continuously for 20 minutes or longer.

11. Inhale the aerosol solution using the prescribed accessory.

12. The device requires no calibration.

13. Do not block the ventilation hole located at the lower part of the unit. It may cause the product to heat up or break, or cause injury.

14. Using the product continuously may heat it up.

CLEANING AND MAINTENANCE

The cleaning of the device must be carried out by using a soft and dry cloth and non-abrasive cleansers.

ATTENTION: During cleaning operations, make sure the internal parts of the unit are not in contact with liquids and that the power plug is disconnected.

CLEANING AND DISINFECTION OF THE ACCESSORIES

19. Carefully follow the cleaning and disinfection instructions of the accessories as they are very important for the device performances and the therapy success.

1. First use and after each treatment:

Disassemble the nebulizer, the mouthpiece, the masks and the nosopiece by using tap water; dip in boiling water for 5 minutes. Reassemble the nebulizer components and connect it to the air outlet (Pic.G), switch the device on and let it work for 10 - 15 minutes.

2. Masks: The masks and air tube must be washed with warm water.

3. Sterilization: Cold disinfecting liquids may be used instead of boiling water, following the manufacturer's instructions.

REFILLING OF THE NEBULIZER

The nebulizer must be replaced after a long period of inactivity, in case it shows deformations or breakages, or when the nebulizer is obstructed by dry medicine, dust, etc. It is recommended to replace the nebulizer after a period between 6 months and 1 year depending on the use. Use original nebulizers only.

REPLACEMENT OF THE AIR FILTER

In normal usage conditions, the air filter must be replaced approximately after 500 working hours or after each year. It is recommended to periodically check the air filter (10 - 12 treatments) and, if the filter shows a grey or brown colour or is wet, replace it. Extract the filter (Pic. A) and replace it with a new one. Do not try to clean the filter for reusing it. The air filter shall not be serviced or maintained while in use with a patient.

REPLACEMENT OF THE ACCESSORY KIT

No. 8 to 14 in the figure are the components. They are available to be purchased separately (model number: AT-GC202). When purchasing any of the above items, contact your local distributor.

△ Use original filters only.

△ Do not use the device without filter.

Possible Problems and How to Solve Them

1. The device does not switch on.

2. The device does not nebulize or nebulizes weakly

SOLUTIONS:

1. Make sure the power plug is firmly fitted to the wall socket;
2. Make sure that the device has been operating within limits indicated in this manual (20 min ON / 40 min OFF);
3. Make sure that the ends of the air tube are tightly fitted to the main unit and nebulizer;
4. Verify that the tube is not empty or has been filled with the proper quantity of medicine (max 8 ml);
5. Verify that the nebulizer nozzle is not obstructed.

Should the device not start working properly again, address to local distributor.

MAINTENANCE and REPAIRS

In case of failure, address to local distributor. Do not open the device in any case. The unit has no user-serviceable parts within and does not need internal maintenance or lubrication.

TECHNICAL SPECIFICATIONS

Model: GC806

Power supply: Please refer to the rating label.

Minimum filling volume: 2 ml

Maximum filling volume: 8 ml

Nebulization rate: 0.35 ml/min (average)

Maximum pressure: 2.2 bar

Weight: 1.5 kg

Size: 130 x 130 x 117 mm

Noise level: 52 dB(A) (average)

Expected service life: 1000 hours

Classification as regards protection against electric shocks.

Nebulizer, mouthpiece and masks are type B applied parts.

Device not protected against splashes.

Device not suitable for use in anesthesia or lung ventilation systems.

NOTE: The technical specifications may change without prior notice.

AEROSOL CHARACTERISTICS ACCORDING TO EN 13544-1 ANNEX CC

Flow: 5.31 l/min

Aerosol output: 0.11 ml

Particle size (MMAD): 3.07 μm

APPLIED STANDARDS

• Electric Safety Standards EN 60601-1

• Electromagnetic Compatibility according to EN 60601-1-2

• Class IIa Medical Device according to the 93/42/EEC European

• Directive on Medical Devices

ENVIRONMENTAL CONDITIONS

Temperature: MIN -20 °C - MAX +60 °C

Humidity: MIN 10 % RH - MAX 95 % RH

Atmospheric pressure: 700 hPa - 1060 hPa

DEVICE SYMBOLS

I ON

O OFF

~ ALTERNATING CURRENT

□ CLASS II EQUIPMENT

■ TYPE BF APPLIED PART

CE0123 COMPLYING WITH THE 93/42/EEC DIRECTIVE

Correct Disposal of This Product (Waste Electrical and Electronic Equipment) (Applicable in the European Union countries and with separate collection systems) This symbol on the product or its literature indicates that the product complies with the electrical and electronic equipment and should not be disposed of with household waste. The user is responsible for disposing the end of life, to the appropriate collection, subject to the penalties provided for in current legislation on waste. For further information regarding the collection systems available contact your local waste disposal service.

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