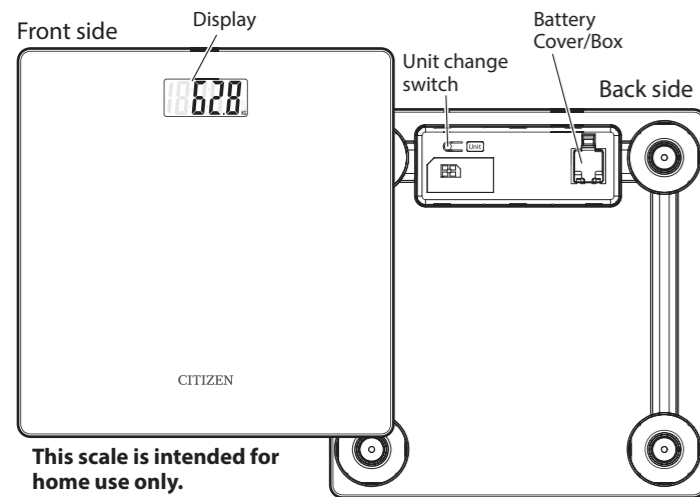
INSTRUCTION MANUAL FOR DIGITAL BODY SCALE HMS324

Refer to instruction manual before use.

Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice.

This device fulfills the provisions of the EMC Directive 2014/30/EU.

* CITIZEN is a registered trademark of CITIZEN WATCH CO., LTD. Japan.

Unit change (KG/LB/ST)

You can switch between units (KG→LB→ST) by pressing the unit change switch [Unit] on the back side while the power is on.

Measuring your Weight

- Place the scale on a solid flat surface before use.
- After moving the scale to a different location or not using it for a long period, place it on a flat and level surface, and then turn the power on (The display shows "0.0") and let it turn off automatically before using it for the first time. This helps to ensure that its weight measurement is accurate.

- Step onto the scale.
Do not move your body until the measurement is complete.
- Check the measurement results.
If the current and previous measurements differ, the difference is displayed for 1 second after the measurement results have been displayed for 5 seconds, and then the scale turns off. If there is no difference, the scale turns off after displaying the measurement results for 5 seconds.

Error Display

Error Display	Cause	Correction
Lo	Battery power is low.	Replace the battery.
---	The weight capacity is exceeded (over 150 kg, 330 lb or 23 st 8 lb).	
E	The weight is unstable.	Measure again.
Err	Measurement is error.	Measure again.

Troubleshooting

Problem	Checks	Corrections
Power does not turn on.	Is the scale placed on uneven ground?	Please place it on a solid flat surface.
	Battery may be running low.	Replace the battery.
Abnormal results are displayed.	Is the battery polarity ⊕ ⊖ placed correctly?	Please put the battery to the right direction.
	Is the scale placed on uneven ground?	Please place it on a solid flat surface.

Maintenance

- If the scale is very dirty, wipe it clean with a cloth moistened with a neutral detergent. Then wipe it with a dry cloth. Never clean the scale with thinner or benzene.
- Do not store the scale in locations exposed to direct sunlight, high or low temperature, high humidity, chemical vapors or excessive amount of dust.
- Do not drop the scale or subject it to other shocks or vibration.
- Do not stand the scale against walls.
- Do not place any objects on the scale.

Specifications

Product Name	Digital Body Scale
Model	HMS324-CN-WH / HMS324-CN-BK
Dimensions	Approx. 280 (L) × 270 (W) × 22 (H) mm
Weight	Approx. 1.05 kg (including battery)
Power	DC 3 V, 1 battery (CR2032)
Battery Life	Approx. 1000 times
Weight Range	KG: 5 kg – 150 kg LB: 11 lb – 330 lb ST: 11 lb – 23 st 8 lb
Minimum Display Unit	KG: 0.1 kg LB: 0.2 lb ST: 0.2 lb
Accuracy	± (1%+1D) D=100 g (5 kg – 150 kg)
Operating Temperature and Humidity	5°C – 35°C / 30%RH – 85%RH
Storage Temperature and Humidity	-20°C – 60°C / 10%RH – 95%RH
Accessories	Instruction manual, a battery (CR2032) for monitor.

* Design and specifications are subject to change without notice.

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2002

SAFETY PRECAUTIONS

WARNING	A potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	A potentially hazardous situation which, if not avoided, may result in minor or moderate injury.

WARNING:

- Keep battery out of the reach of children and infants.
- In case of battery ingestion, please follow doctor's directions immediately.
- In case the battery fluid gets in your eyes or your body touches the battery fluid, rinse with plenty of water and consult a doctor immediately.
- Do not dispose of a battery in fire.
- Do not open the battery.
- Do not use the scale on slippery surfaces, such as a wet floor.
- Do not step on the edge area of the scale.
- Do not jump on the scale.

CAUTION:

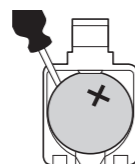
- Do not use the scale when your feet are wet. Keep dry the glass surface of the scale to avoid slipping and falling down.
- Do not attempt to disassemble the scale.
- Do not store in the locations that are exposed to chemicals or corrosive vapours.
- Do not place the scale on cushioned floor surface such as on a carpet or a mat.
- Remove the battery if the scale is left unused for a long period.

Battery Placement

Before using the scale first time, pull out the battery insulation film.

* When "Lo" is displayed, replace the battery with a new one.

- Open the battery cover.
- Insert a thin tip into the point shown in the figure, and remove the old battery using the thin tip.
- Put a new 3 V lithium battery (CR2032) into the battery box with the ⊕ polarity facing up.
- All digits on the display show up instantly. Then, the display will return to "0.0".

* When disposing of used battery, comply with governmental regulations or environmental public institution rules that apply in your region.

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