CITIZEN

# Instruction Manual

DIGITAL BLOOD PRESSURE MONITOR

Wrist Measuring  
Automatic Model

# CH-617

English  
Español

# Contents

- 01..... **B**lood Pressure Monitor Intended Use
- 02..... **1**. What is blood pressure ?
- 02..... **2**. Why is it useful to measure blood pressure at home ?
  - ..... A. WHO blood pressure classifications
  - ..... B. Variations in blood pressure
- 05..... **3**. Important information before use of the device
- 08..... **4**. Device description
  - ..... A. Name of the parts
  - ..... B. Description of display symbols
- 10..... **5**. Changing Batteries
- 11..... **6**. Attaching pressure cuff
- 12..... **7**. Correct measurement position

# Contents

- 13..... **8.** Symbol Difications.
- 15..... **9.** How to measure.
  - ..... A. Inserting batteries in position.
  - ..... B. Setting date and time.
  - ..... C. Steps to take blood pressure measurement.
  - ..... D. Storing recalling and erasing measurement data.
  - ..... E. To change batteries.
- 18..... **10.** Maintenance.
- 19..... **11.** Safe-keeping.
- 19..... **12.** To prevent malfunction.
- 20..... **13.** Warning.
- 21..... **14.** Specifications.

## **MEDICAL DISCLAIMER**

This manual and product are not meant to be a substitute for advice provided by your doctor or other medical professional.

You are not to use the information contained herein, or this product for diagnosing or treating a health problem or prescribing any medication. If you have or suspect that you have a medical problem, promptly consult your healthcare provider.

## **Blood Pressure Monitor Intended Use:**

Measures human's Systolic blood pressure, Diastolic blood pressure and pulse on wrist by oscillometric measurement method.

The device is designed for home use and recommended for use by adults aged 18 years and older with wrist circumference ranging 135 ~ 195 mm (5.3~7.7 inch).

## **1. What is blood pressure ?**

Blood pressure is a measurement of the force of blood flowing against the walls of the arteries. Arterial blood pressure is constantly changing during the course of the cardiac cycle. The highest pressure in the cycle is called the systolic blood pressure, the lowest is the diastolic blood pressure. Both pressure readings, the systolic and the diastolic are necessary to enable a physician to evaluate the status of a patient's blood pressure. Many factors such as physical activity, anxiety or the time of day, can influence your blood pressure. Blood pressure is typically low in the morning and increases from the afternoon to the evening. It is on average lower in the summer and higher in the winter.

Medical practitioners generally recommend the “Rule of 3”, where you are encouraged to take your blood pressure three times in a row (at 1 minute interval), three times a day for three days. After three days you can average all the results and this will give you an accurate idea of what your blood pressure really is.

## **2. Why is it useful to measure blood pressure at home ?**


Having one's blood pressure measured by a doctor in a hospital or a clinic, and a group health checks, tend to stimulate nervousness in the subject and may even create high blood

pressure. Also varies blood pressure in accordance with a variety of conditions and so judgment is not possible on the basis of a single measurement.

The blood pressure measured first thing in the morning after getting up, before taking any food and with the subject still, is known as the fundamental blood pressure. In practice it is rather difficult to record the fundamental blood pressure, but to come as near as possible to measuring the blood pressure in an environment that is close to this, is why it is useful to take the measurement at home.

## A. WHO blood pressure classifications

Standards for assessment of high or low blood pressure without regard to age, have been established by the World Health Organization (WHO), as shown in the chart.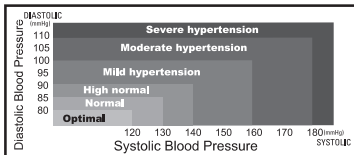

\* Figure No.1


This chart is intended to be used as a guide in understanding noninvasive blood pressure measurements. Please consult with your physician for proper diagnosis.

## B. Variations in blood pressure

Individual blood pressures vary greatly both on a daily and a seasonal basis. These variations are even more pronounced in hyper tense patients. Normally the blood pressure rises while at work and is at its lowest during sleeping period.

**(Hyper tense: means a person who has high blood pressure symptom.)**

The graph below illustrated the variations in blood pressure over a whole day with measurement taken every five minutes. The thick line represents sleep. The rise in blood pressure at 4 PM (A in the graph) and 12 PM (B in the graph) correspond to an attack of pain.

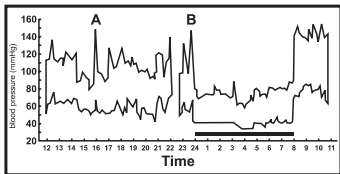(Direct arterial pressure recording in unrestricted man. Beven, Honour & Stott: Clin. Sci. 36:329. 1969)

\* Figure No.2

### 3. Important information before use of the device

- \* Do not use this manual and product as a substitute for advice, diagnosing or treating a health problem or prescribing any medication by your doctor.
- \* The device is not suitable for clinical use.
- \* This monitor is not intended for use in the MR environment.

- ⊙ Please rest for at least 5~10 minutes before taking the measurement.
- ⊙ To allow your blood vessels to return to the condition prior to taking the measurement, please wait at least 3~5 minutes between measurements. You may need to adjust the wait time according to your personal physiological situation.
- ⊙ If you have one of the circulatory problems as arteriosclerosis, diabetes, liver disease, kidney disease, severe hypertension, peripheral circulation and medical problem, please consult your doctor or healthcare professional before using the device.
- ⊙ Wait 30 ~ 45 minutes before measurement if you've just consumed caffeinated beverages or smoked cigarettes.
- ⊙ Blood pressure measurements should be interpreted by a physician or trained health professional who is familiar with your medical history. By using the unit regularly and recording the results for your physician to interpret, you keep your physician informed of the continuing trends in your blood pressure.
- ⊙ Wind the cuff snugly around your wrist and the cuff must be at the same level as your heart.

- ⊙ Do not vibrate the unit during measurement, or the proper measurement will not be achieved.
- ⊙ Perform measurement quietly in a relaxed position.
- ⊙ Do not wind the cuff over jacket or sweater sleeve, or measurement cannot be done.
- ⊙ Keep in mind, that blood pressure naturally varies from time to time throughout the day and also is affected by lots of different factors such as smoking, alcohol consumption, medication and physical activity.
- ⊙ People with a condition that causes circulatory problems (diabetes, kidney disease, arteriosclerosis or poor peripheral circulation) may get lower readings with this monitor that with a blood pressure monitor that is used on the upper arm. Please consult your physician to determine if your wrist blood pressure accurately reflects your actual blood pressure.
- ⊙ Blood pressure measurement determine with the unit are equivalent to those obtained by a trained observer using the cuff / stethoscope auscultation method. Within the limits prescribed by the Standard of EN 1060-4.
- ⊙ We recommend you using the same wrist (preferably the left wrist) and measuring around the same time each day.
- ⊙ Perform measurements in a quiet and relaxed environment at room temperature.
- ⊙ This product is not suitable for:
  - ⊙ Pregnant women
  - ⊙ People with arrhythmias
  - ⊙ Undergoing intravenous injection on any limb
  - ⊙ Currently in a dialysis treatment
  - ⊙ In pre-eclampsia condition


- ⦿ For those who have had mastectomy surgery (especially whose' lymph nodes removed), it's recommend take a measurement on the unaffected side.
- ⦿ When used among medical electronic equipments on the same limb, pressurization of the cuff may cause temporarily malfunction to other devices.

**\*Note!**

1. Do not use the device on infants, children, or those who cannot express their own intention.
2. The device is equipped with sensitive electronic components. While measuring, avoid strong electrical or electromagnetic fields, e.g. mobile phones, microwave ovens, etc; or it may lead to temporary reading error or inaccuracy.
3. Consider the electromagnetic compatibility of the device (ex. power disturbance, radio frequency interference etc.) Please use it indoor only.
4. Over high frequency measurements may result in blood flow interference, which is likely to cause uncomfortable sensations, such as partial subcutaneous hemorrhage, or temporary numbness to your wrist. In general, these symptoms should not last long. However, if you do not recover in time, please seek your medical practitioners for help.

## 4. Device description:

### A. Name of the parts

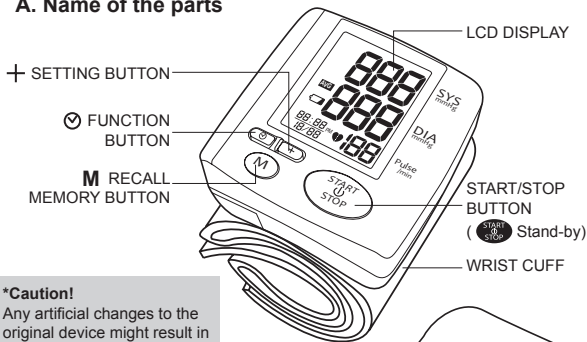
#### \*Caution!

Any artificial changes to the original device might result in measurement error. Substitution of a component different from that supplied might result in measurement error.


ALKALINE BATTERY  
UM-4 (AAA) LR03 1.5 V x 2

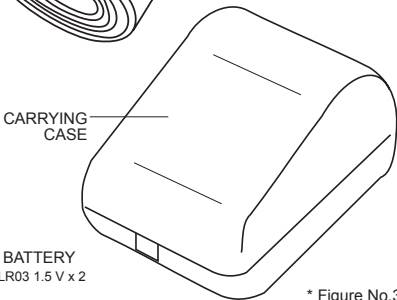
\* Figure No.3

## B. Description of display symbols
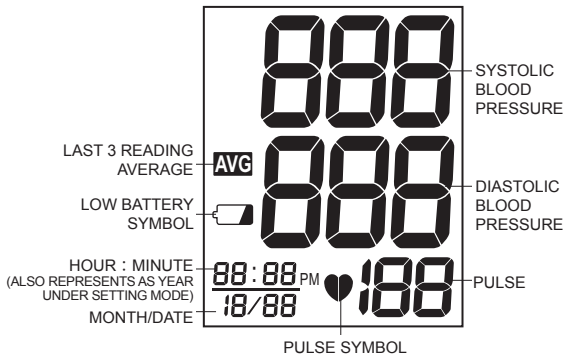
## 5. Changing batteries

Remove the battery cover at the left side of the unit and insert batteries into the battery compartment in accordance with ribbon strip as shown, taking extreme care that the polarities + and - are observed. Important notice: to take the batteries out easily, just pull the ribbon strip for help.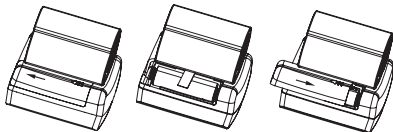

\* Figure No.4

Remove the battery cover and insert 2 AAA LR03 alkaline batteries into the battery compartment as shown.


### **\*Note!**

- Batteries are hazardous waste. Do not dispose of them together with the household garbage. Please take the used batteries to the recycling collection point according to your local regulations.
- Keep the battery away from small children in case they swallow it.
- To prolong the battery life and prevent damage caused by leakage, remove the batteries from the device if the device is not to be used for a long period.
- Memories (if any) will not be deleted during battery replacement.
- After replacing the batteries, reset the date and time.

## 6. Attaching pressure cuff

A. Wrap the pressure cuff around the wrist.


- The display of the unit should be placed on the palm side of the wrist.
- The wrist should be bare.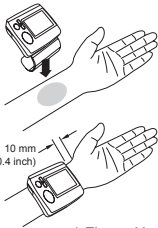

\* Figure No.5

B. Fasten the pressure cuff snugly.

- Do not pull strongly on the pressure cuff.
- Do not make the pressure cuff too tight.
- Leave approximately 10 mm (0.4 inches) between the sleeve and the bottom of the palm of your hand.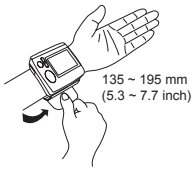

\* Figure No.5-1

### \*Note!

- Do not use this device if your wrist has any wound or injury.
- Do not wrap the cuff around any body part other than your wrist.


## 7. Correct measurement position

It is extremely important for the measured values, that the cuff is at the same height of the heart.

Higher or deeper position will give deviations from the true value.

### Measurement while sitting down:

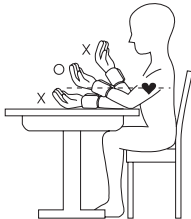- Sit down comfortably with your feet flat on the floor.  
Please do not cross your legs during measurements.
- Position the blood pressure monitor on your wrist.
- Place your elbow on a table or other object.
- Use the armrest to position the arm so that the pressure cuff is at the same height as the heart.
- Relax your hand with the palm facing up.

\* Figure No.6

## 8. Symbol Definitions

SYMBOLS	Definitions
 <b>Low Battery Symbol</b>	<p>This symbol appears when the battery power is excessively low or the polarity reverses.</p> <p>→ We suggest you replace all batteries with new ones, and make sure the +/- polarities are properly positioned.</p>
 <b>Pulse Symbol</b>	<p>Once pulse is detected, the symbol flashes with each pulse beat.</p> <p>→ Our suggestion: Please do not talk or move during measurements.</p>
 <b>Last 3 Reading Average</b>	<p>This symbol appears when LCD displays average value of last 3 readings.</p>

SYMBOLS/ SYMPTOMS	CONDITIONS/ CAUSES	INDICATION/ CORRECTION
<b>No power-on by pressing</b>  <b>button</b>	Worn-out batteries.	Replace them with 2 new AAA (LR03) alkaline batteries.
	Battery polarities have been positioned incorrectly.	Re-insert the batteries in the correct positions.
<b>EE</b> <b>Measuring Error Symbol shown on display or the blood pressure value is displayed excessively low or high.</b>	Cuff has been placed incorrectly.	Wrap the cuff properly so that it is positioned correctly.
	Did you talk or move during measurement?	Measure again. Keep wrist steady during measurement.
	Shaking of the wrist with the cuff on.	
<b>E1</b> <b>Measuring Error Symbol</b>	Air circuit abnormality. Cuff tube may not be plugged into monitor correctly.	Check cuff connection. Measure again.
<b>E2</b> <b>Measuring Error Symbol</b>	Inflation pressure exceeding 300 mmHg.	Switch the unit off, then measure again.
<b>E3</b> <b>Measuring Error Symbol</b>	Can't determine blood pressure measurement data.	Wrap the cuff properly and keep steady. Measure again.
<b>Note:</b> If "EP" appears on the display, just return the device to your local distributor or importer.		

## 9. How to measure

### A. Inserting batteries in position:

- Open battery cover.
- Refer to the picture in positioning batteries. (Figure No.4)
- All LCD segments appear on the display in 3 seconds.

### B. Setting Date and Time:

- Press "  " button over 2 secs ("year" starts flashing)  
Press " + " button to set the correct year (2011,2012,...2026)
- Press "  " button ("month" starts flashing)  
Press " + " button to set the correct month (1,2,3,.....12)
- Press "  " button again ("date" starts flashing)  
Press " + " button to set the correct date
- Press "  " button again ("hour" starts flashing)  
Press " + " button to set the correct hour in 12-hour format
- Press "  " button again ("minute" starts flashing)  
Press " + " button to set the correct minute (0,1,2,3,.....59)
- Press "  " button again to switch to normal time  
(month, date, hour and minute appear on the display)

### C. Steps to take blood pressure measurement:

- Wrap the cuff around the wrist (Refer to Wrapping of Wrist cuff as Figure No. 5)
- Sit upright on the chair to have correct posture. (Refer to Correct Posture in Taking Blood Pressure as (Figure No. 6)
- Press "  " button. All elements display '888' in 3 seconds. Afterward, displays the updated time. Pulse symbol will flash

on the display and then the air pressure will automatically pump up to 195 mmHg. If blood pressure tends to be on the high level, the air pressure will automatically increase to the required level for measurement.

**\*Note!**

Do not inflate the cuff until it is wrapped around your wrist.

- \* Do not move or talk in the midst of taking blood pressure measurement.
- d. After the air pressure is increased, it will slowly decrease. When the pulse is detected,  will start flashing.

**\*Note!**

- If the cuff does not stop inflating, remove the cuff at once.
- To interrupt the measurement, you may press the  button. The cuff will deflate immediately after a button is pressed.

- e. After taking blood pressure measurement, the Systolic blood pressure, Diastolic blood pressure and pulse will be on the display for 1 minute.

**D. Storing, recalling and erasing measurement data:**

- a. Storing data:

After each blood pressure measurement, the Systolic blood pressure, diastolic blood pressure, pulse and the time & date of specific day will be automatically stored. The memory holds the latest 99 measurement data. If more than 99 measurements, the memory will automatically clear out the earliest data.

b. Recalling data:

- (1). Press " M " button. If there is no data in the memory, only TIME /DATE will appear on the display. If yes, the first reading will be the average of the latest 3 measurements. (The average value dose not appear if there are two or fewer measurements stored in memory.)
- (2). Press " M " button again, the latest measurement will appear with the number as well as the recording date and time. The data called up on the display is numbered accordingly. After reading the recalled data as rotator 99-set of the stored data, press " M " to switch to normal time.
- (3). To discontinue in the midst of reading of the data, press "  " button.

c. Erasing data:

- (1). Press the "M" button.
- (2). Press hold " ☺ " and " + " button to clear memories. "CLA" show up on the display for 3 seconds.

**Note: Once deleted, your data can NOT be restored.**

**E. To change batteries:**

When "  " show up on the display, the battery is weak. It will not be possible to take blood pressure measurement. All batteries used must be the same type. Do not mix alkaline, standard(carbon-zinc) or rechargeable (cadmium) batteries. Do not mix old and new batteries. Please replace both batteries together.

## 10.MAINTENANCE:

- ⦿ Use a piece of cloth with water or mild cleansing agent to wipe the device and dry it immediately with a dry cloth.
- ⦿ Do not use detergent or any strong chemicals to clean the device.
- ⦿ Disinfection - Use a piece of cloth with a dilution of neutral detergent or soapsuds to wipe the surface of the cuff for 10 seconds.
- ⦿ Make sure the cuff is completely dry before using.
- ⦿ If any suggestion or service is requested, please consult your local distributor.
- ⦿ Do not implement the maintenance procedures for device during measurement.  
Only trained technicians are allowed to repair and disassemble the device, including software upgrades, patches and maintenance.

### **\*Note!**

Water quality required for cleaning and disinfection: Tap water.

## **11.SAFE-KEEPING:**

- ⦿ Always keep the device in the carrying case after its use.
- ⦿ Do not put the item directly under the sunlight, in high temperature, or humid and dusty places.
- ⦿ Do not store the device in extremely low (less than  $-13^{\circ}\text{F}$  /  $-25^{\circ}\text{C}$ ) and high (more than  $158^{\circ}\text{F}$  /  $70^{\circ}\text{C}$ ) temperature, nor in a place its humidity exceeds 93% R.H.

## **12.TO PREVENT MALFUNCTION:**

- ⦿ Do not in any way twist the wrist cuff.
- ⦿ Do not start the air pressure when the wrist cuff is not wrapped around the wrist.
- ⦿ Do not attempt to disassemble or change any parts of the monitor including wrist cuff.
- ⦿ Do not drop the product or put it through strong impact.

## **13.WARNING:**

- ⊙ The device measures the blood pressure of adult only.
- ⊙ In the event of the cuff pumping up and not stopping, please open the cuff at once.
- ⊙ To ensure precise measurement reading, recalibration of the device is recommended after 2 years from the date of purchase. Shipping plus handling cost and recalibration service fee shall be charged accordingly.

### **Limited Warranty**

This device is warranted to be free from manufacturing defects for a period of 1 year from the date of purchase. Within the warranty period, should there be any manufacturing defect found and repair is needed, please contact your local distributor accordingly.

Please note warranty does not cover damage caused by, including but not limited to: misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repair or modification; improper use of electrical/ power supply; dropped product; failure to follow required maintenance and safe-keeping; transportation damage.

## 14. SPECIFICATIONS:

Model Number	:CH-617
Method of Measurement	:Oscillometric
Range of Measurement	:Pressure 0 ~ 300 mmHg Pulse 40 ~ 199 beats/minute
Accuracy	:Pressure +/- 3 mmHg, Pulse +/- 5% of reading
Inflation	:Automatic inflation (Air Pump)
Deflation of Pressure	:Automatic air release control valve
Display	:Liquid Crystal Digital Display
Sets of Memory	:99 sets
Unit Dimension	:L70XW72XH31mm L2.75 x W2.83 x H1.22 inch
Unit Weight	:88 ± 5 g (3.10 ± 0.17 oz) (Batteries and cuff excluded)
Cuff Size	:Wrist circumference approx. 13.5 - 19.5 cm
Storage/Transportation Environment	:Temperature: -25 °C ~ 70 °C (-13 °F ~ 158 °F) Humidity: ≤ 93 % R.H.
Operation Environment	:Temperature: 5 °C ~ 40 °C (41 °F ~ 104 °F) Humidity: 15 % ~ 93 % R.H. Atmospheric pressure: 700 hPa ~ 1060 hPa
Auto power off	:Whenever not used for 1 minute
Product Life	:5 Years (4 times per day)
Power Supply	:2 x AAA LR03 (2 x 1.5 V) Alkaline battery (3V DC)
Battery Life	:Approx. 250 times measurement
Accessories	:Carrying case, instruction manual, 2 pcs AAA LR03 battery

These specifications are subject to change without notice for purpose of improvement.

## Note:

This blood pressure monitor complies with the EC Directive (93/42/EEC) and bears the CE mark. This blood pressure monitor also complies with mainly following standards,(included but not limited)

### Safety standard:

EN60601-1 Medical electrical equipment Part 1: General requirements for basic safety and essential performance

### EMC standard:

EN 60601-1-2 Medical electrical equipment Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests

### Performance standards:

EN 1060-1 Non-invasive sphygmomanometers - Part 1: General requirements

EN 1060-3 Non-invasive sphygmomanometers - Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems

EN 1060-4 Non-invasive sphygmomanometers - Part 4: Test procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers

EN ISO 81060-1 Non-invasive sphygmomanometers - Part 1: Requirements and test methods for non-automated measurement type (partially applied)

IEC 80601-2-30 Medical electrical equipment - Part 2-30: Particular requirements for the basic safety and essential performance of automated non-invasive sphygmomanometers

Follow instructions for use.

Date of Manufacture

---

### **TYPE BF APPLIED PART**

- Internally powered equipment
  - Not suitable for use in presence of flammable anaesthetic mixture with air or with Oxygen or nitrous oxide
  - Continuous operation with short-time loading
- 

## **IP22** Degrees of protection provided by enclosures

---

To avoid inaccurate results caused by electromagnetic interference between electrical and electronic equipments, do not use the device near a cell phone or microwave oven. At least keep a maximum output power of 2 W yields and a distance 3.3m away from this equipment.

---

Discard the used product to the recycling collection point according to local regulations.

---

Manufacturer:  
HEALTH & LIFE Co., Ltd.  
9F, No. 186, Jian Yi Road, Zhonghe District, New Taipei City,  
Taiwan  
[www.healthandlife.com.tw](http://www.healthandlife.com.tw)

---


Authorized Representative in the European Community:  
EMERGO EUROPE  
Prinsessegracht 20, 2514 AP The Hague, The Netherlands

**SN**


Serial Number

**LOT**


Batch code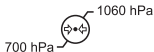

Atmospheric pressure limitation

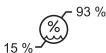Humidity limitation

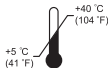Temperature limitation

## WEEE MARK

If you want to dispose this product, do not mix with general household waste. There is a separate collection systems for used electronics products in accordance with legislation under the WEEE Directive (Directive 2012/19/EU) and is effective only within European Union.

## Information for Users on Collection and Disposal of used Batteries.

The symbol in this information sheet means that used batteries should not be mixed with general household waste.

For proper treatment, recovery and recycling of used batteries, please take them to applicable collection points.

For more information about collection and recycling of batteries, please contact your local municipality, your waste disposal service or the point of sale where you purchased the items.

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- CITIZEN is a registered trademark of Citizen Watch Co., Ltd. Japan.
## **CITIZEN SYSTEMS JAPAN CO., LTD.**